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The RAN interface with speech is intended for the connection of a third-party patient unit with call function and speech connection to the room bus (RAN).

A third-party patient unit model can be used only after approval by Tunstall GmbH.

Dimensions (HxWxD): 13 x 51 x 95 mm (without fixing clips)

Weight: 28 g (without fixing clips)

**Note!** The complete installation of the system is described in the Technical Manual.

**Caution! Electrostatic sensitive components**

The printed circuit board contains components at risk from electrostatic charges. Therefore, avoid touching it.


**Caution!** During installation the power supply should be switched off to ensure zero-potential status. Otherwise, the RAN interface could be damaged.

**Mounting**


Clip the RAN interface with both fixing clips onto a top-hat rail, e.g. in the medical supply unit, see Fig A.

**A RAN interface with speech**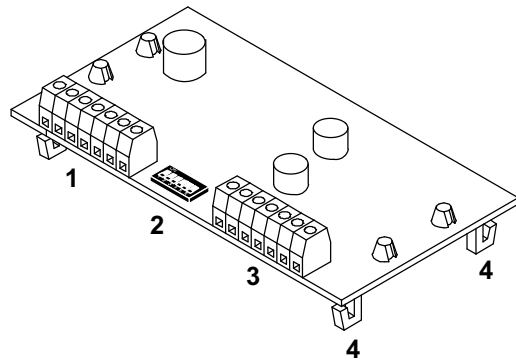

- 1 Connector for room bus (RAN)
- 2 DIP switch
- 3 Connector for third-party system
- 4 Two fixing clips for mounting on a top-hat rail

**B Connections**

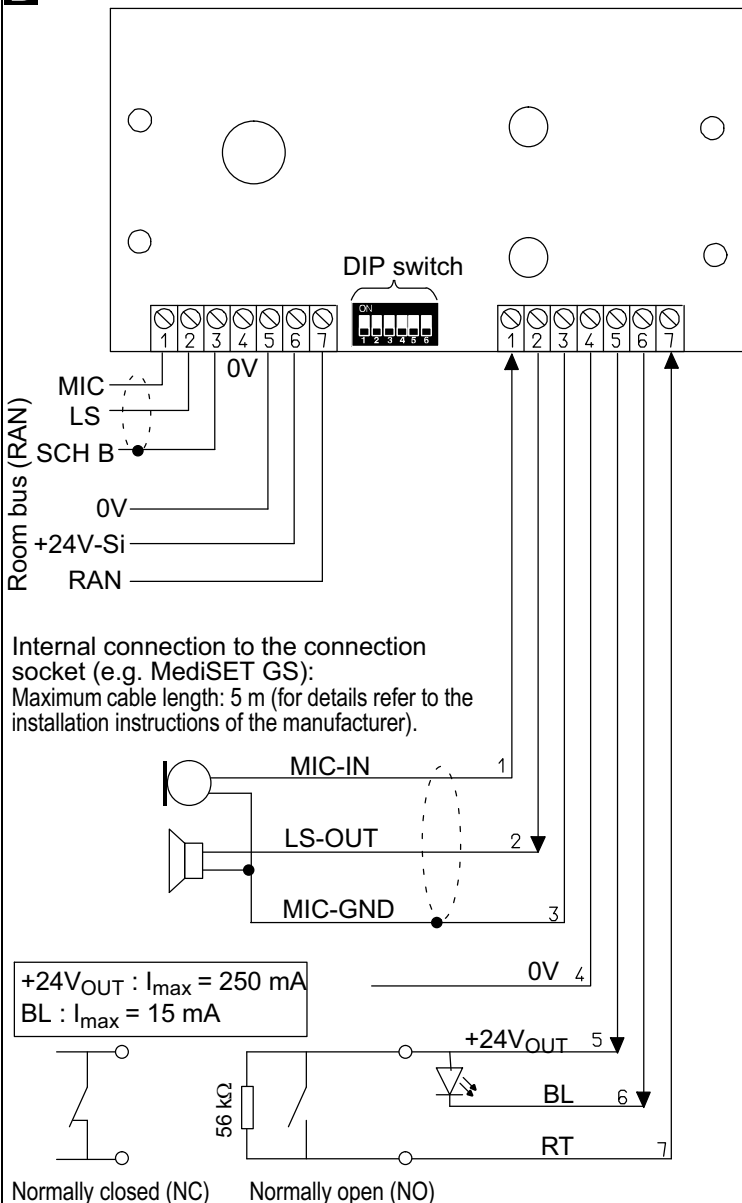
**C Setting the DIP switch**


**DIP switch 1-3:**

Setting the bed number: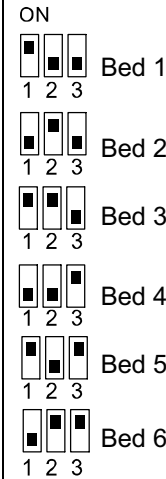


**DIP switch:**

Contact type of the connected call button: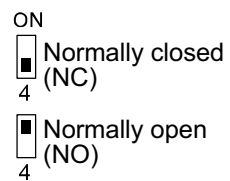

**DIP switch 5-6:**

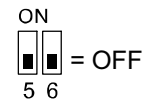**Attention!** A normally open contact (NO) can be hard-wired or plug connected. A normally closed contact (NC) has to be hard-wired. A plug connection is not allowed.

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Utilisation d'un terminal de patient d'un système externe seulement après autorisation par Tunstall GmbH.