## DE - Installationsanleitung

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## EN - Installation Instructions

### 77 0181 10: Room lamp universal, 3 sections, with doorplate

for signalling of all call types (red), presence of staff 1 (green) as well as additional display for WC call (white). Doorplate as label field for room designation.

### 77 0181 00: Room lamp universal, 4 sections, with doorplate

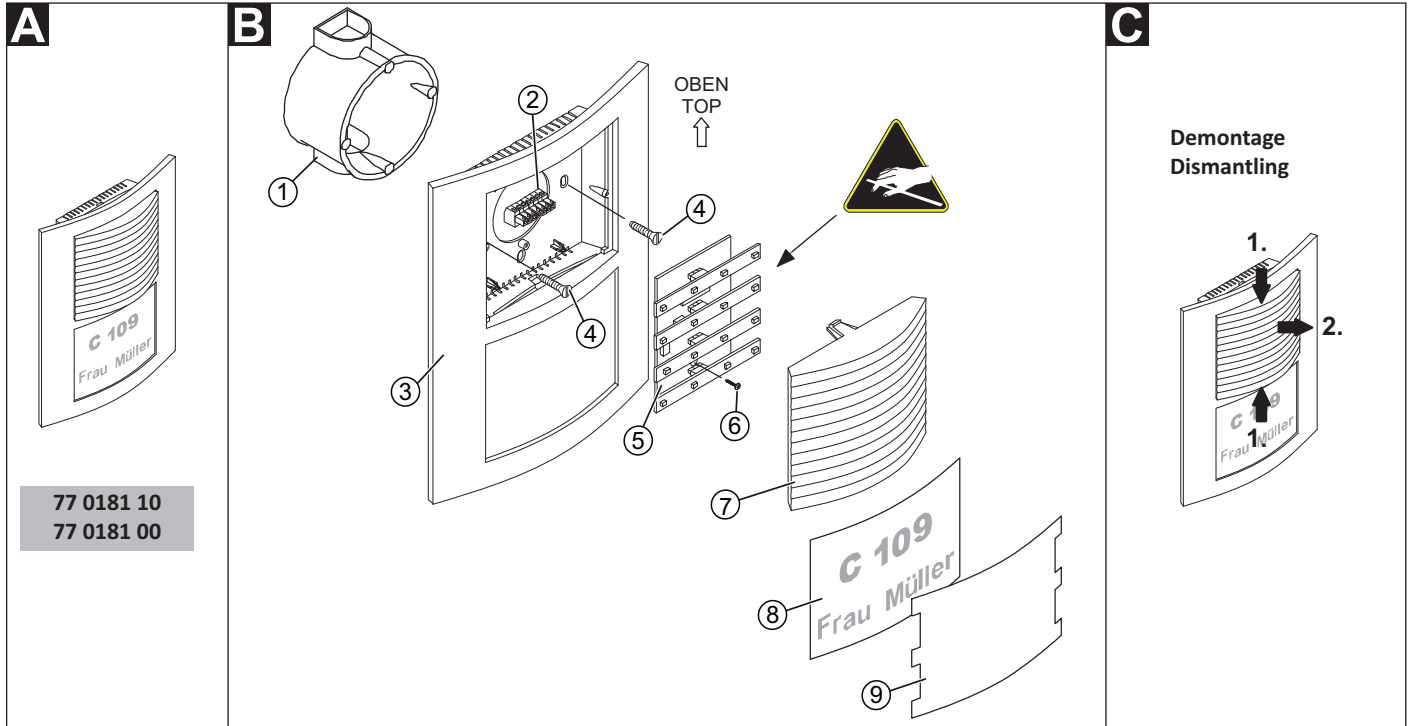
as 77 0180 10, but additionally presence of staff 2 (yellow).

**NOTE!** The complete installation of the system is described in the Technical Manual.

**CAUTION!** The LED modules include electrostatic sensitive components. Avoid touching.

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- |  |  |
|--|--|
| [1] *Back box, 1-gang<br>(solid wall: 17 0100 00,<br>partition wall: 17 5100 00) | [5] PCB with LED modules                         |
| [2] *Connector, 7-pole<br>(70 0807 07)   | [6] Fastening screw for the PCB                  |
| [3] Housing  | [7] Light dome with insert module                |
| [4] *Back box screws   | [8] *Label strip (height x width:<br>70 x 92 mm) |
|  | [9] Protection cover                             |

\* Not included in the scope of delivery, please order separately.

### B Mounting

At the point of delivery the room lamp is assembled and must be dismantled as follows, see fig. D:

- Lightly press the light dome [7] together from top and bottom. Then pull off the light dome [7].

Wall installation on a one-gang back box [1] fixed with screws on the lateral holes of the back box, see fig. B:

1. Prepare the wires and connect them to the connector [2] (see section "Connection").
2. Plug the connector [2] from the rear onto the PCB [5] (in the rear of the housing [3]).
3. Bolt the housing [3] to the back box [1] with the screws [4]. The arrow on the rear of the housing points toward the top.
4. Press the light dome with insert module [7] onto the decorative frame.
5. Insert the label strip [8] and the protection cover [9].

To mount the PCB on the vertical holes of the back box, the PCB must be removed temporarily:

1. Loosen the fastening screw [6] of the PCB and remove the PCB (first bottom, then top).
2. The PCB is installed in the reverse order.

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**Connection**

1. Strip the connection cable in the back box to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
2. Connect the wires to the connector [2] according to the connection diagram.
3. If required: Plug the LED module onto the PCB (on the rear of the housing) according to section „LED module colours“.

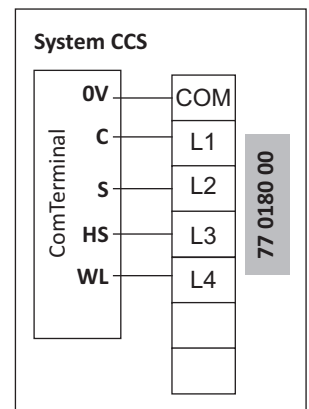
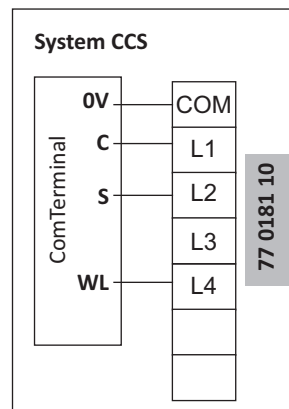
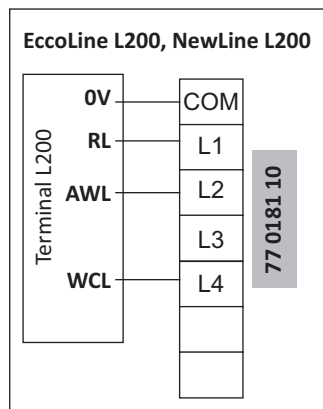
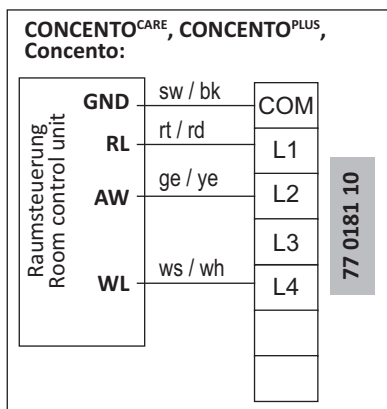
**Note:** The colour of an LED module can be identified by a colour spot of the same colour on the surface of the LED module.

**LED module colours**

	Connection point	77 0181 10	77 0181 00
	L1	Red: Calls	Red: Calls
	L2	Green: Staff 1	Green: Staff 1
	L3	No LED module	Yellow: Staff 2
	L4	White: WC call	White: WC call

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## EN - Installation Instructions

**77 0181 10: Room lamp universal, 3 sections, with doorplate** for signalling of all call types (red), presence of staff 1 (green) as well as additional display for WC call (white). Doorplate as label field for room designation.

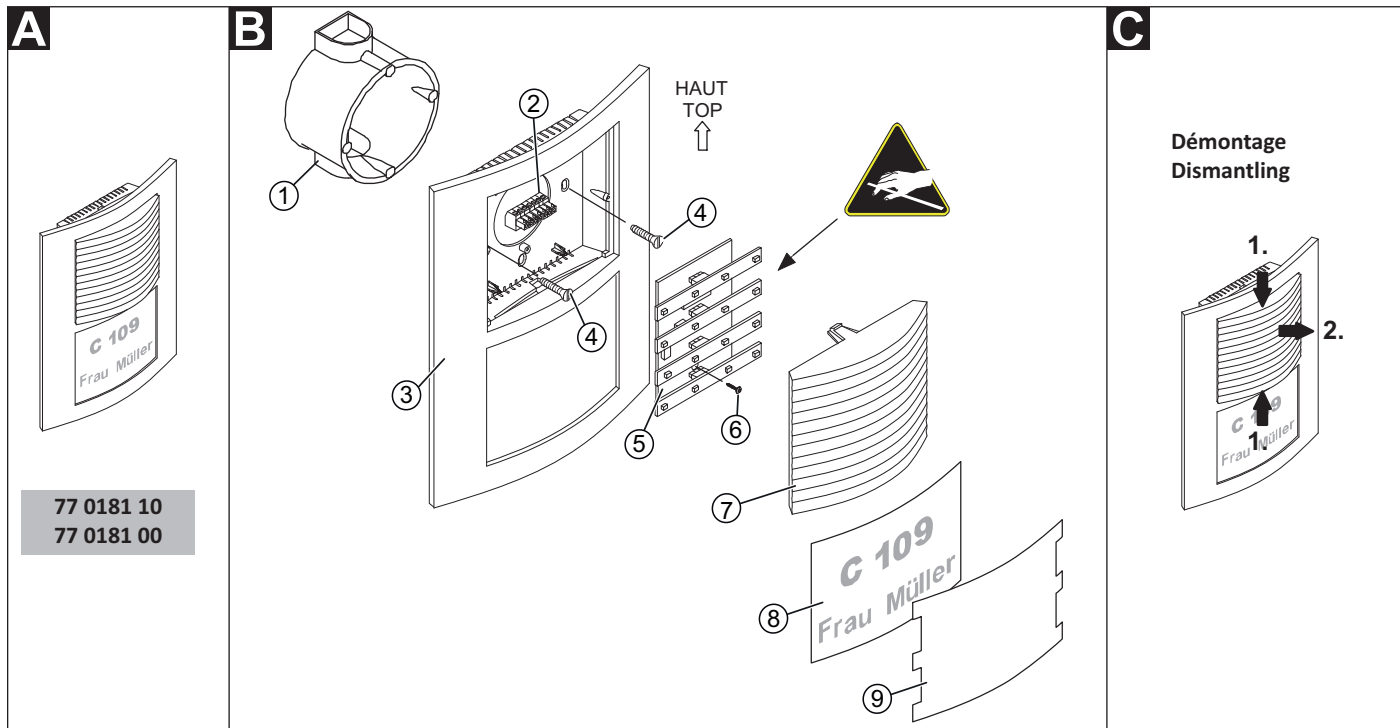
**77 0181 00: Room lamp universal, 4 sections, with doorplate** as 77 0180 10, but additionally presence of staff 2 (yellow).

**NOTE!** The complete installation of the system is described in the Technical Manual.

**CAUTION!** The LED modules include electrostatic sensitive components. Avoid touching.

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- |  |   |
|--|---|
| [1] *Back box, 1-gang (solid wall: 17 0100 00, partition wall: 17 5100 00) | [5] PCB with LED modules                      |
| [2] *Connector, 7-pole (70 0807 07)  | [6] Fastening screw for the PCB               |
| [3] Housing  | [7] Light dome with insert module             |
| [4] *Back box screws   | [8] *Label strip (height x width: 70 x 92 mm) |
|  | [9] Protection cover                          |

\* Not included in the scope of delivery, please order separately.

### B Mounting

At the point of delivery the room lamp is assembled and must be dismantled as follows, see fig. D:

- Lightly press the light dome [7] together from top and bottom. Then pull off the light dome [7].

Wall installation on a one-gang back box [1] fixed with screws on the lateral holes of the back box, see fig. B:

- Prepare the wires and connect them to the connector [2] (see section "Connection").
- Plug the connector [2] from the rear onto the PCB [5] (in the rear of the housing [3]).
- Bolt the housing [3] to the back box [1] with the screws [4]. The arrow on the rear of the housing points toward the top.
- Press the light dome with insert module [7] onto the decorative frame.
- Insert the label strip [8] and the protection cover [9].

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## EN - Installation Instructions

To mount the PCB on the vertical holes of the back box, the PCB must be removed temporarily:

1. Loosen the fastening screw [6] of the PCB and remove the PCB (first bottom, then top).
2. The PCB is installed in the reverse order.

### Connection

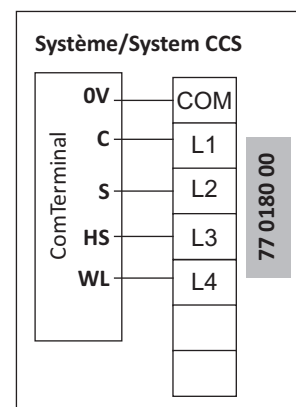
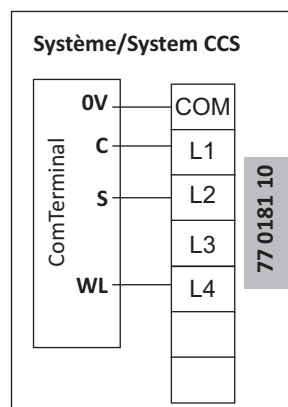
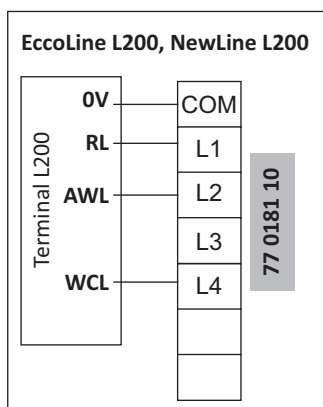
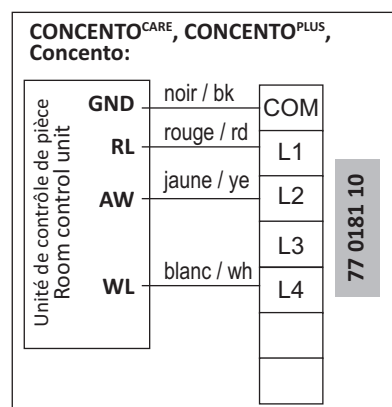
1. Strip the connection cable in the back box to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
2. Connect the wires to the connector [2] according to the connection diagram.
3. If required: Plug the LED module onto the PCB (on the rear of the housing) according to section „LED module colours“.

**Note:** The colour of an LED module can be identified by a colour spot of the same colour on the surface of the LED module.

### LED module colours

Connection point	77 0181 10	77 0181 00
L1	Red: Calls	Red: Calls
L2	Green: Staff 1	Green: Staff 1
L3	No LED module	Yellow: Staff 2
L4	White: WC call	White: WC call

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