Steckvorrichtung Kombi, TVL
Best.-Nr. 70 0424 50, 70 0425 50

Connection socket combi, TVL
order no. 70 0424 50, 70 0425 50

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Intended use

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Connection socket combi, TVL order no. 70 0424 50, 70 0425 50

A connection socket with two different sockets for connection of patient units and call devices. Designed for the following functions: transmission of calls, control of two lights, TV transmission, transmission of entertainment programmes, blinds control. The use of the supported functions depends on the connected patient unit or call device. TV connection e.g. via IR TV control module universal Audio (70 0360 11). Suitable for wall mounting.

 **NOTE!** The complete installation of the system is described in the Technical Manual.

 **CAUTION!** The printed circuit board includes electrostatic sensitive components. Avoid touching.

Mounting (→ fig. B)

1. Fit the socket [1] into the back box.
2. Screw the mounting frame [2] together with the socket [1] onto the back box.
3. Connect both plugs [3] with the sockets (plug A into socket A, plug B into socket B).
4. Push the cover [4] onto the mounting frame [2].

Setting DIP switches (→ fig. C)

Setting of bed numbers 1 to 6: Use DIP switches "P3", "P2", "P1" as shown in the table of figure C.

Example: Bed number 1 = "P3" OFF, "P2" OFF, "P1" ON


* Note: When using bed number 6 no diagnostic call can be used.

Setting DIP switch "TV" for use Blinds control:

- DIP switch "TV" OFF: No blinds control connected.
- DIP switch "TV" ON: Blinds control.


DIP switch "PG" must always be set to OFF.

A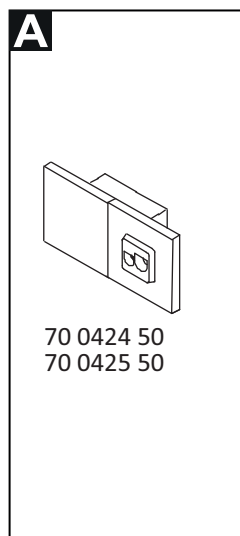


70 0424 50
70 0425 50

B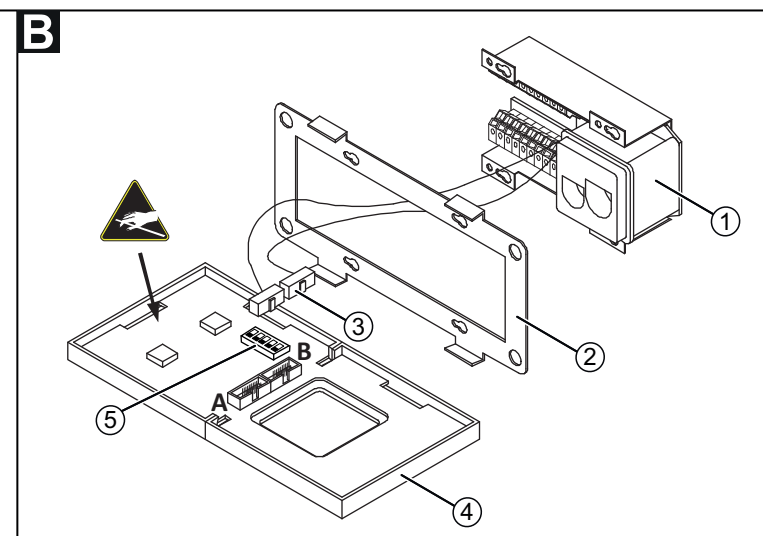

C

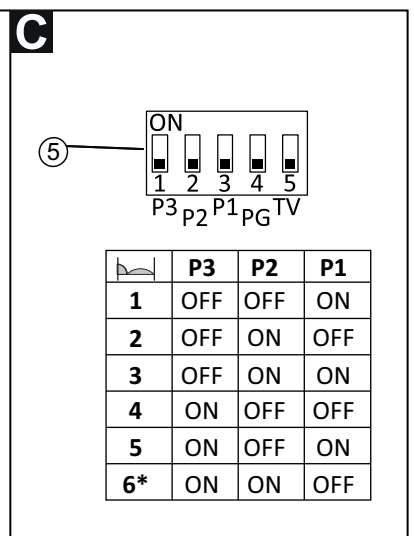	P3	P2	P1
1	OFF	OFF	ON
2	OFF	ON	OFF
3	OFF	ON	ON
4	ON	OFF	OFF
5	ON	OFF	ON
6*	ON	ON	OFF

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Notes on the connections (→ fig. D, E)**Handling of the push wire connectors**

Usable wire gauges: 0.55 - 1 mm Ø.

Connect only one wire per opening.

1. Strip the wire (8 mm).
2. Insert the stripped wire into the terminal until it stops.
3. For detaching the wire insert a screw driver with a tip of 2.5 x 0.4 mm or 3.5 x 0.5 mm into the opening and pull out the wire.

Shield wire

For connecting the shield wire, use the screw driver to hold the spring contact open, while you are inserting the shield wire. Afterwards check for a firm wire contact by pulling at the wire.

Room bus (RAN)

- All cables for speech transmission (LS, MIC, SCH B) may be only stripped to a maximum of 30 mm to maintain the shielding function.
- Insulate the drain wire (SCH B) to prevent short circuiting.

Light control

The switching outputs LT1 for reading light and LT2 for room light provide an output voltage of 24 V DC and an output current of max. 200 mA, sourced from the ComTerminal power supply. The potential refers to the 0 V connections of the nurse call system. The outputs prepare a switching pulse (24 V DC, max. 200 mA) for as long as the button on the patient unit is held down (ePat^{®lite}, pear push switch, or patient handset).

TV

The available TV functions depend on the TV set in use. Before the final installation, technicians shall check with the ordering party for the project to clarify the details regarding the connection and the functions.

Technical data

Power Supply	24 V DC
Standby current consumption	30 mA
Dimensions (HxWxD)	90 x 181x 45 mm
Housing colour 70 0424 50	RAL 9018 papyrus white
Housing material 70 0424 50	ABS
Housing colour 70 0425 50	similar to RAL 9016 studio white
Housing material 70 0425 50	Polycarbonate
Degree of protection	IP20
Ambient temperature	+5°C – +40°C
Relative humidity	0 % – 85 %

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© Tunstall GmbH, Orkotten 66, 48291 Telgte, Germany, www.tunstall.de