Funkempfänger-T, Bestell-Nr. Z 00 8202 31

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Radio receiver-T, order no. Z 00 8202 31

Plug-in connection to:

Call switch with connection socket: 19 0704 00, 10 0704 00

Call switch with 2 connect. sockets: 19 0704 20, 10 0704 20

Radio receiver-T, order no. Z 00 8202 32

Plug-in connection to:

Connection socket L200: 73 0455 00 (rev. 4), 73 0400 00

Connection socket L200 bedhead unit: 73 045 00 (rev. 4)

Radio receiver-T, order no. Z 00 8202 33

Plug-in connection to:

Connection socket combi: 70 0425 00, 70 0424 00

Connection socket combi, TVL: 70 0425 50, 70 0424 50

Connection socket combi universal: 70 0449 00

Conn. socket combi, bedhead unit: 70 0435 00, 70 0434 00

C. socket combi, bedhead unit, TVL: 70 0434 50, 70 0435 50

C. socket combi, bedhead unit, universal: 70 0448 00

Connection socket with call switch: 70 0171 60A,
70 0171 60C, 70 0171 60F, 70 0171 03, 70 0171 00

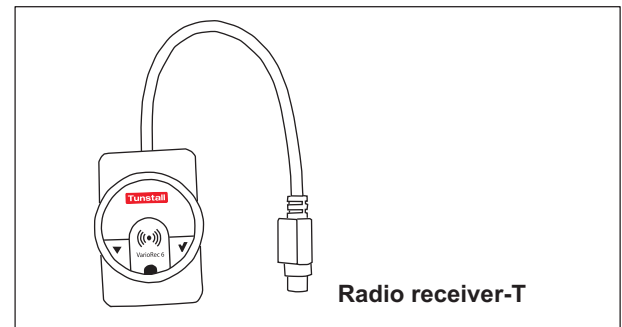
Connection socket with call switch, bedhead unit: 70 0171 50

Connection socket call devices: 70 0400 00

Radio receiver-T, order no. Z 00 8202 36

Plug-in connection to:

Call switch insert with connector: 29 0704 00...

Radio receiver-T

The radio receiver-T is able to receive signals from the following radio transmitters:

- Radio trigger including call button: MyAmie (P68007/02, P68007/04), see page 3.
- Radio trigger incl call button and fall detection: iVi™ (P68005/47)
- Universal sensor (61005/30)
- Wireless step-on sensor mat (Z00800301, Z00800302)
- Radio sensor mat 869 MHz (Z00800201)
- Large surface radio pneumatic switch (75071100)
- Radio smoke detector (68005/96)

The use of further radio transmitters is possible. For this you have to contact Tunstall GmbH.

The radio receiver-T is plug connected to a connection socket of the nurse call system in the socket for pear push switch. Activating the radio transmitter will raise the call in the nurse call system that would be raised by a pear push switch in the same socket.

NOTE! The radio transmission is not monitored. That's why the radio transmitters may only be used as additional call devices in the nurse call system.

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Check radio coverage on the location of use

The radio range between the radio receiver-T and the radio transmitters depends on the structural conditions of the building; the range is up to 30 m. Before handing over a handheld radio transmitter (MyAmie, iVi) to the user, you must check the radio coverage at the place of use. To do this, walk around the planned place of use and trigger a call over and over again. Inform the user of the radio coverage.

Documents for VarioRec6 / radio receiver-T

The radio receiver-T is an OEM product of Lehmann Electronic GmbH. In addition to these installation instructions, you must read and observe the following documents from Lehmann Electronic:

- Brief overview VarioRec6 radio receiver (supplied with the radio receiver-T), document no. LE235
- Operating instructions VarioRec6, document no. LE243

The product designation VarioRec6 is a collective term of Lehmann Electronic for various radio receivers. The radio receiver-T is only one of these. Not all topics in the VarioRec6 documents apply to the radio receiver-T. Therefore, please observe the following notes:

Available functions

The following functions mentioned in the documents for VarioRec6 are available on the radio receiver-T with the Tunstall radio transmitters:

- Teaching in radio transmitters
- Teaching out radio transmitters
- Teaching out all taught-in radio transmitters
- Acknowledge fault message
- Set master mode
- Care mode
- Set operating display active / inactive
- Reset all parameters to factory settings
- Daily message monitoring
- Vital monitoring

Functions not available

The following functions mentioned in the documents for VarioRec6 are **not** available on the radio receiver-T with the Tunstall radio transmitters:

- Radio transmitter for special functions, e.g. radio cancel switch or radio light switch

Error/failure messages during operation

When used with Tunstall radio transmitters, the radio receiver-T displays the following error and fault messages mentioned in the documents for VarioRec6:

- Transmitter battery low
- System fault
- Reception blockade
- Daily report missing
- Vitality message is missing

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Information about low battery level

The radio transmitters contain batteries. When the battery level of a radio transmitter is low, the LED on the radio receiver-T will flash in red to inform the nursing staff, that the battery or the radio transmitter must be changed. Please refer to the documents for the VarioRec6.

ATTENTION! The nursing staff must identify and acknowledge a low battery level by watching the LED on the radio receiver-T and/or the signals on the radio transmitter. Radio transmitters with a low battery level must be replaced immediately.

Note: If the radio transmitters are used with a social alarm unit a call is raised to a monitoring centre in case of a low battery level. This call is **not** raised when the radio transmitters are used with the radio receiver-T!

LED on the MyAmie, order no. P68007/02, P68007/04

The MyAmie is used to raise calls by pressing the call button. The LED on the MyAmie confirms, that the button is pressed, and indicates the battery level at the same time:

- LED is on (for approx. 3 seconds) after the call button has been pressed: Battery is okay. A call is raised.
- LED is flashing after the call button has been pressed: The battery level is low. A call is raised. The LED on the radio receiver-T is flashing red to indicate, that the battery is low and the MyAmie must be changed.
- LED remains dark after the call button has been pressed: The battery is empty or the MyAmie is defective! No call will be raised! The MyAmie must be changed immediately.

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