**RAN-Multiplexer Flamenco, Best.-Nr. 77 0270 00**

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RAN-Multiplexer Flamenco, Best.-Nr. 77 0270 00

**RAN multiplexer Flamenco, order no. 77 0270 00**

Multiplexer controls the nurse call functions of up to 5 rooms. Connection to the group bus (OSYnet), the room bus (RAN), and the 24 V DC power supply of the nurse call system via connection socket ComTerminal.

The connection socket ComTerminal (order no. 70 0491 00) is not included in the scope of delivery of the RAN multiplexer.

**NOTE!** The basic installation of the nurse call system is described in the Technical Manual.

**1. Product characteristics**

- Max. 5 rooms can be connected
- Max. 50 room users (switches, connection sockets, room lamps, etc.) can be connected, distributed as needed among the 5 rooms and the collective display (as of software revision 1.01)
- A room lamp can be used as a collective display (display of messages from all connected rooms) (as of software revision 1.01)
- Total max. 100 m room bus cable (RAN)
- A failure of the collective display is indicated in the nurse call system as a fault of room 1.

**2. Technical data**

Dimensions (HxWxD):	205 x 110 x 34 mm
Power supply:	24 V DC
Standby current consumption without connected RAN users:	38 mA
Housing material:	ABS
Ambient temperature:	+5 °C – +40 °C
Relative humidity:	0% – 85% (non condensing)

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### 3. Preparatory work for renovations

**NOTE!** This chapter 3 does not apply to a new installation of a RAN multiplexer Flamenco.

The RAN multiplexer Flamenco, order no. 77 0270 00, makes it possible to switch over a room bus (RAN), which was connected to a CCS bus via a RAN multiplexer, order no. 70 0270 00, to a group bus (OSYnet).

The installed connection socket ComTerminal, via which the RAN multiplexer (70...) was connected, can still be used for the new RAN multiplexer Flamenco.

To prepare for the update you need to:

- write down the address of the old RAN multiplexer (70...)
- count the number of connected RAN users in each room.

#### 3.1 Dismantling the old RAN multiplexer (70...) > Fig. A

1. To remove the lower cover [6] of the RAN multiplexer [5], press a screw driver into a lateral recess and lever out the cover [6].
2. Write down the RAN multiplexer's address, coded by the 10-pole DIP switch [7], as described in the chapter 3.2 and shown in Fig. B.
3. Remove the RAN multiplexer [5] from the mounting plate [3] by releasing it at the snap lug and simultaneously pushing it upwards.
4. Disconnect the flat cable from the connection socket [2].
5. Unscrew the four fixing screws [4] until the mounting plate [3] comes loose. Remove the mounting plate [3]. The screws [4] remain in the connection socket [2].

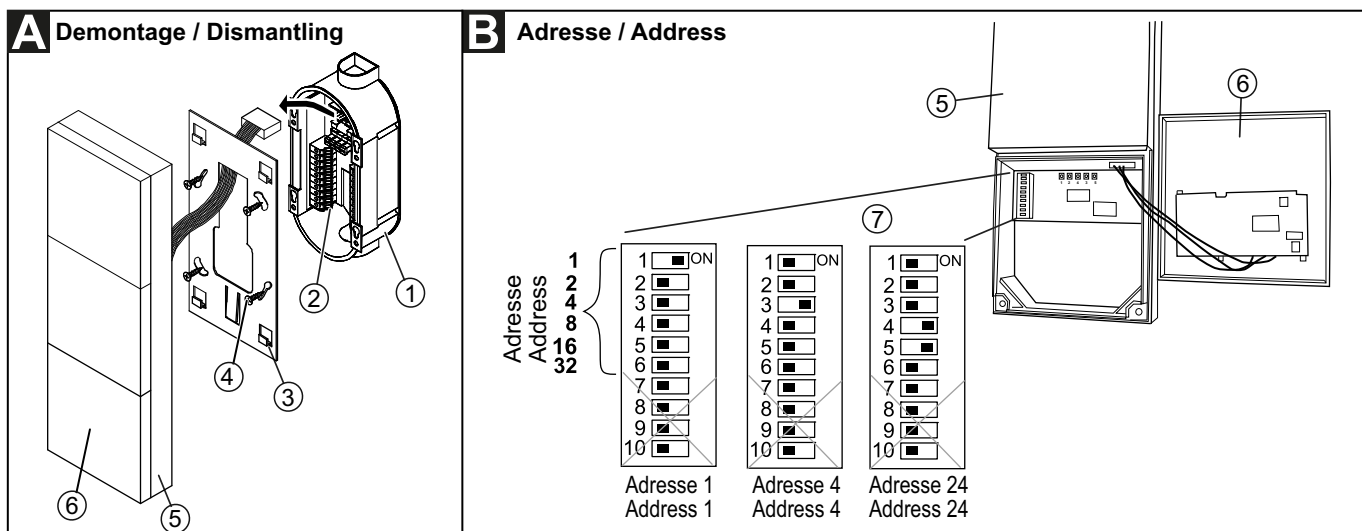
Do not dismantle the connection socket [2]. It will be reused with the new RAN multiplexer Flamenco.

#### 3.2 Address of the old RAN multiplexer (70...) > Fig. B

The RAN multiplexer's address is coded by DIP switches 1 to 6 of the 10-pole DIP switch [7].

DIP switch 1 codes address 1. DIP switch 2 codes address 2, DIP switch 3 codes address 4 etc, see Fig. B. The appropriate DIP switches are set to „ON“. Other addresses are coded by addition, e.g. DIP switch 4 + DIP switch 5 = Address 24.

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4. Connecting the connection socket > Fig. C

1. Connect the wires of the connection cables and the RAN expander [9] to the connection socket [2] according to Fig. C.
2. Insert fine wire fuse 2 A/F 5x20 [8] into the fuse holder of the connection socket [2].

**ATTENTION, fuse!** The connection socket (70 0491 00) is delivered with a fuse 1 A/F. For operation with the RAN multiplexer this fuse must be replaced by a fine wire fuse 2 A/F 5x20 (00 0130 36).

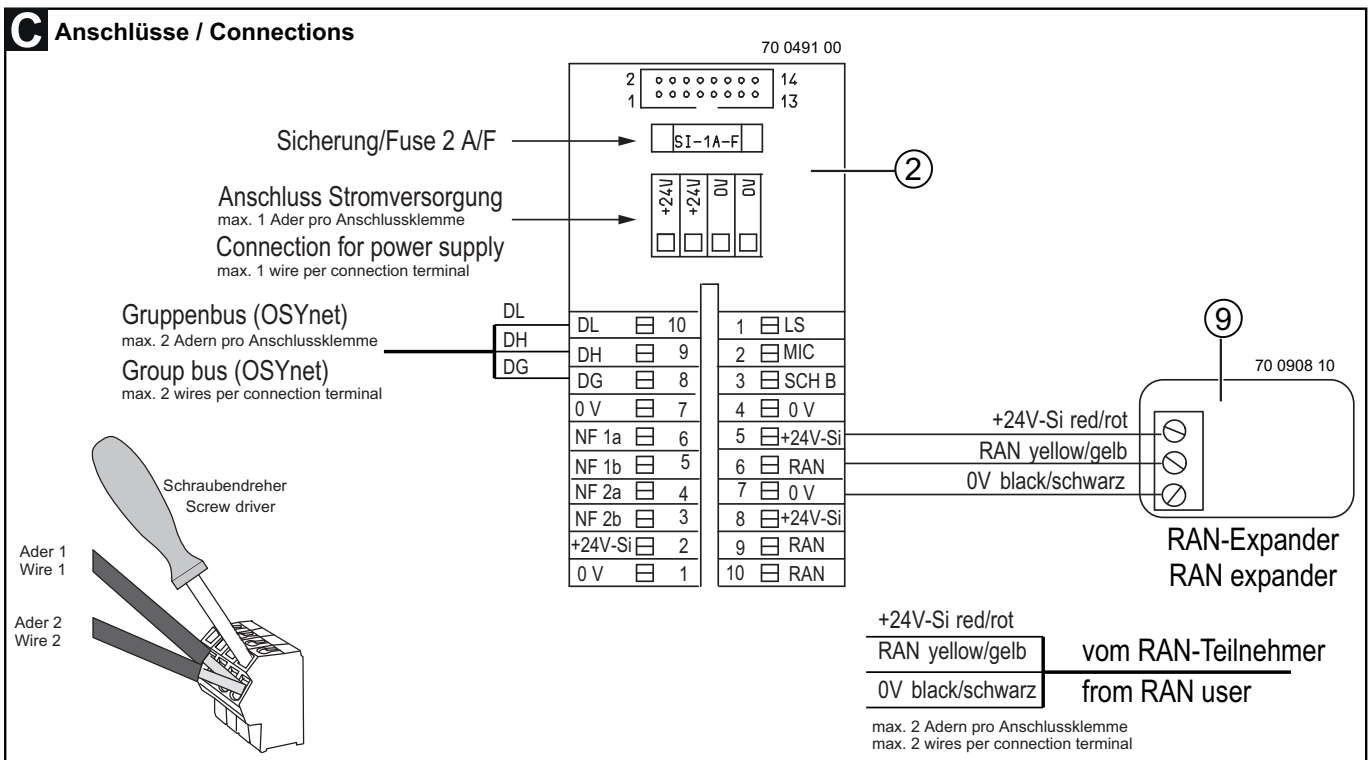
**ATTENTION, terminating resistor!** In the last device on the group bus (OSYnet) an 120 ohms resistor (00 0040 76) must be fitted between the connection points DH and DL.

Handling of terminals with solderless connection technique

Prerequisite: The wires must be stripped to 8 mm.

- Connect a wire: Insert the stripped wire into the terminal until it stops. To make insertion easier put a small screw driver into the opening (see Fig. C) while inserting the wire.
- Disconnect a wire: For loosen the wire put a small screw driver into the opening (see Fig. C) and pull out the wire.

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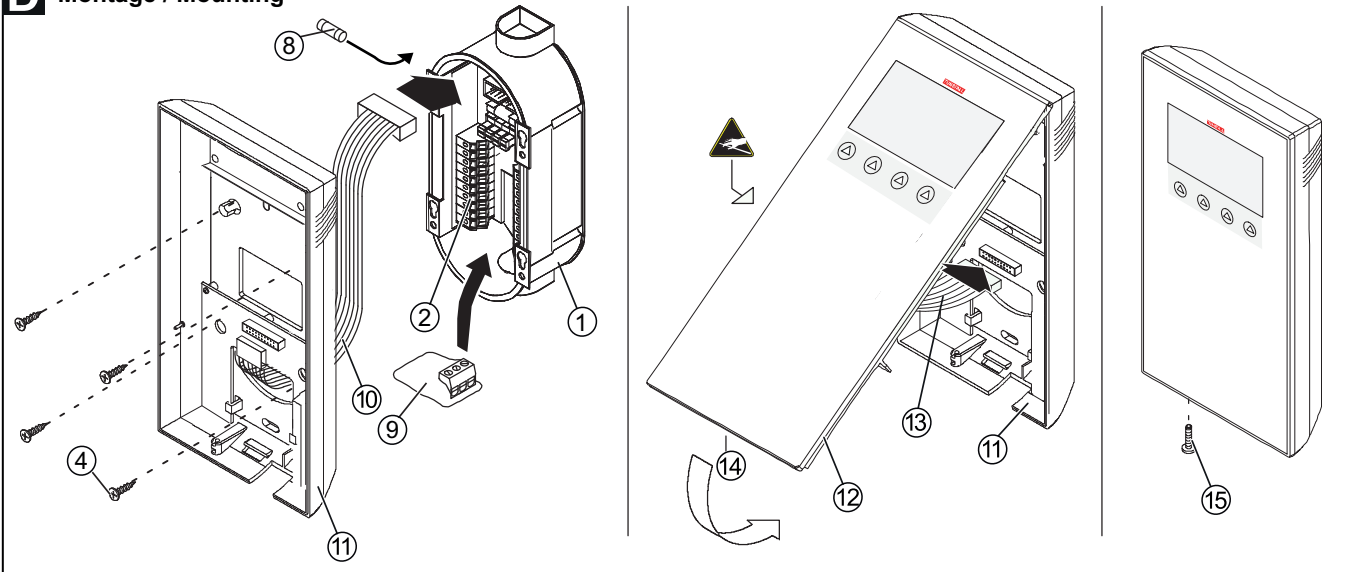
## 5. Mounting &gt; Fig. D

1. Connect the cables to the ComTerminal connection as described in chapter 4.
2. Insert fine wire fuse 2 A/F 5x20 [8] into the fuse holder of the connection socket [2] as described in chapter 4.
3. Loosen the four back box screws [4].
4. Insert the flat cable [10] through the outlet in the housing rear panel [11] and plug it into the appropriate socket on the connection socket [2].
5. Fix the housing rear panel [11] using the back box screws [4] to the back box [1].
6. Plug the flat cable [13] of the housing front panel [12] into the appropriate socket on the housing rear panel [11].
7. Place the housing front panel [12] onto the top of the housing rear panel [11] creating a hinge. Now press the housing front panel [12] downwards onto the rear panel [11] with low pressing power until the catch [14] engages. Make sure that the flat cable [13] is not pinched.
8. For improved security a securing screw [15] can be fitted into the existing hole.

Finally, the RAN multiplexer Flamenco has to be configured, see „Configuration Instructions“ as of page 6.

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- |  |  |
|--|--|
| 1 - * Back box                                   | 10 - Flat cable of the housing rear panel  |
| 2 - * Connection socket ComTerminal (70 0491 00) | 11 - Housing rear panel                    |
| 4 - * Back box screws                            | 12 - Housing front panel                   |
| 8 - Fuse 2 A/F (00 0130 36)                      | 13 - Flat cable of the housing front panel |
| 9 - RAN expander                                 | 14 - Catch                                 |
|  | 15 - Securing screw                        |

\* Not included in the scope of delivery of the RAN multiplexer Flamenco.

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**6. Room addresses and channel numbers**

A maximum of 5 rooms can be connected to one RAN multiplexer. The RAN multiplexer converts the channel numbers of its connected room devices (call switches, connection sockets etc.) to room addresses, see Fig. E.

The room address of the first connected room must be set at the RAN multiplexer. The four following addresses, i.e. the addresses of the other rooms, will be assigned automatically.

The room addresses are always assigned in steps of five, even if only three rooms are connected to the RAN multiplexer. The unused room addresses remain blank.

When renovating, set the same values for room addresses and channel numbers that were set in the old installation.

**7. Set channel numbers at room devices**

Room devices = Switches, connection sockets, room lamps etc. **No** pear push switch!

- Set the channel number at each room device.

Every device in a room must have the same channel number.

The five rooms connected to a RAN multiplexer are numbered from 1 to 5, i.e. channel number 1 to 5.

NOTE! At connection sockets the channel number equals to the bed number.

Refer to the installation instructions delivered with the room device to learn how to set its channel number or bed number.

**Set channel number for a collective display**

It is possible to use a room lamp as a collective display, i.e. it signals the calls and presences of all 5 rooms connected to the RAN multiplexer Flamenco.

- To set a room lamp as a collective display, the channel number 0 must be set.

NOTE! Channel 0 must not be set on any other devices (switches, connection sockets, etc.).

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**NOTE!** Usually, one room lamp corresponds to one RAN user. In the former Tunstall product range, there were room lamps that unusually corresponded to 2 RAN users. This is noted in the documentation for the respective room lamps.

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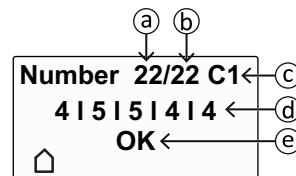
### 8.4 Checking the room bus RAN

#### 8.4.1 Status

Check whether the set RAN numbers for room devices and collective display equal to the numbers of operational devices, see “8.2.3 Setting of RAN numbers” and “8.2.4 Setting RAN number for collective display” .

1. Mark „RAN“ using the arrow keys. Then press the check-mark.
2. Mark „Status“ using the arrow keys. Then press the check-mark.

**NOTE!** The Test can take some time.

- Ⓐ Sum of all operational room devices
- Ⓑ Sum of all set room devices
- Ⓒ C1: Collective display (1 RAN user) operational  
C2: Collective display (2 RAN users) operational
- Ⓓ Operational room devices per room  
Room 1 | Room 2 | Room 3 | Room 4
- Ⓔ „OK“ : All set RAN numbers (room devices, collective display) match to the numbers of operational devices.

„Error“: **Not** all set RAN numbers (room devices, collective display) match to the numbers of operational devices! Fix error.

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- Press the home key to end. In case of an **error** perform a „Test RAN“.

**8.4.2 Test RAN**

Test if all devices (room devices, collective display) are operational and if they are correctly connected to the RAN.

1. Mark „RAN“ using the arrow keys. Then press the check-mark.
2. Test start: Mark „Test RAN“ using the arrow keys. Then press the check-mark.

**HINWEIS!** The Test can take some time.

The LEDs of all connected devices must flash. Non-flashing devices are wrongly connected or defective. Replace defective devices. Correct any wrong connection.

3. Compare the number of flashing room devices with the set RAN number for room devices. If the set RAN number is wrong, correct the RAN number setting after the RAN test as described on the previous page.
4. End of test: Press the home key.

**8.5 Exit the configuration menu**

When all settings are made and all tests are completed, exit the configuration menu:

- Press home key several times until normal operation display appears.

**NOTE!** If for a period of three minutes no function key is pressed, the system will leave the configuration menu.

**9. Configuring the nurse call system (SystemOrganizer)**

The complete nurse call system must be configured using the PC software „SystemOrganizer“. How to configure a nurse call system with the SystemOrganizer is described in the online help for the SystemOrganizer.

For configuring the RAN multiplexer Flamenco within the SystemOrganizer you need to know the following additional information:

For each RAN multiplexer Flamenco 5 locations must be created. For each of these 5 locations the following properties must be set:

- Location Type: Patient Room
- Installed Device: RoomTerminal
- The 5 locations must be in the same ward and have 5 consecutive addresses.

**NOTE!** Even if less than 5 rooms are connected to the RAN multiplexer Flamenco, 5 locations must be created in the SystemOrganizer.

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#### 4. Connecting the connection socket > Fig. C

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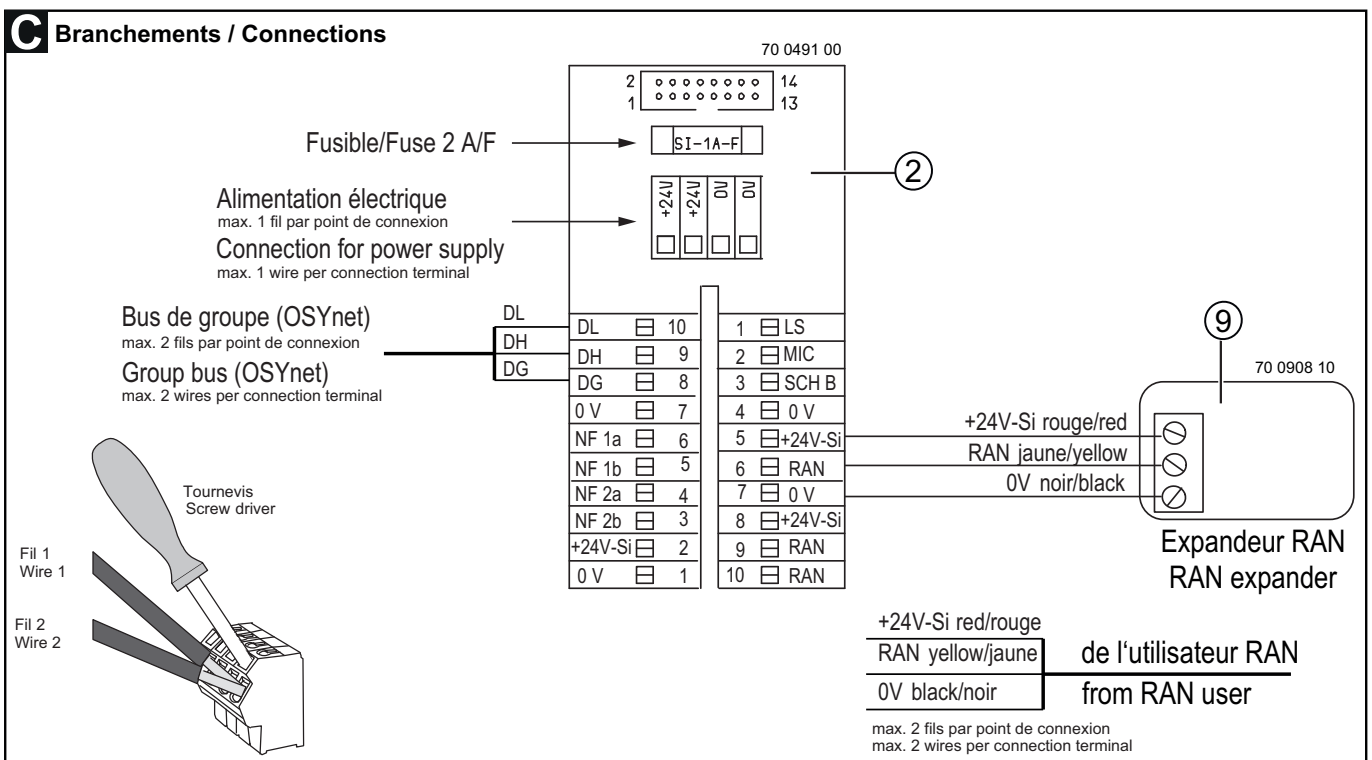
**ATTENTION, terminating resistor!** In the last device on the group bus (OSYnet) an 120 ohms resistor (00 0040 76) must be fitted between the connection points DH and DL..

#### Handling of terminals with solderless connection technique

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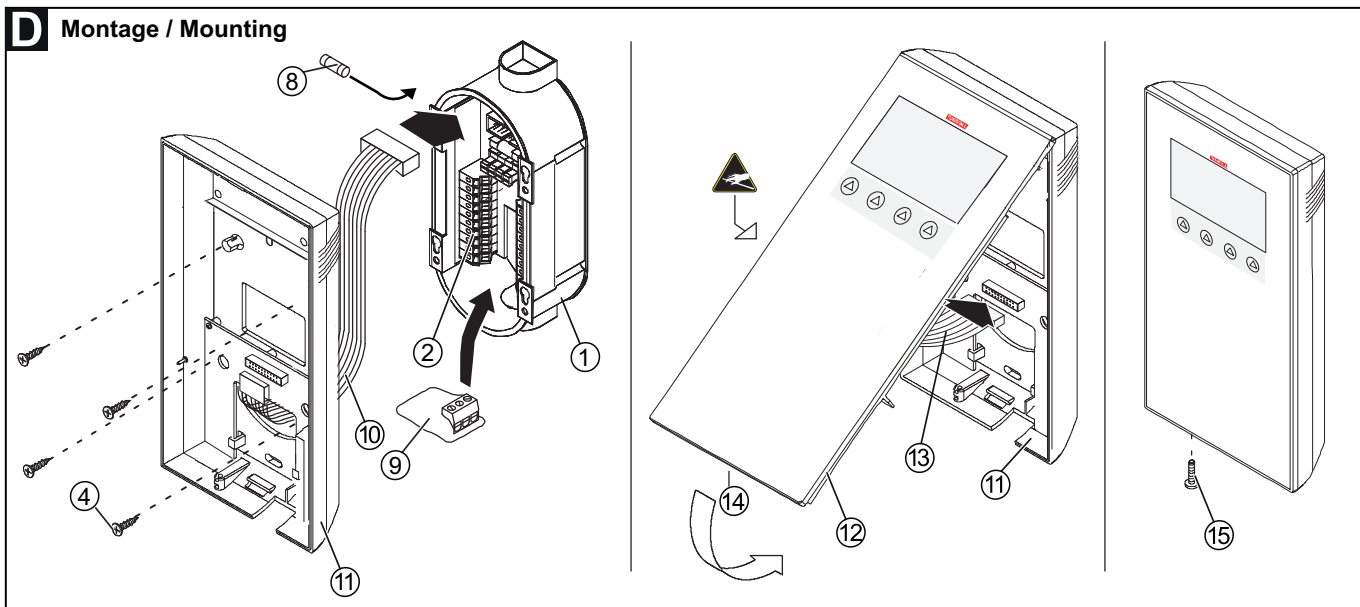
Finally, the RAN multiplexer Flamengo has to be configured, see „Configuration Instructions” as of page 6.

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- |  |  |
|--|--|
| 1 - * Back box                                   | 10 - Flat cable of the housing rear panel  |
| 2 - * Connection socket ComTerminal (70 0491 00) | 11 - Housing rear panel                    |
| 4 - * Back box screws                            | 12 - Housing front panel                   |
| 8 - Fuse 2 A/F (00 0130 36)                      | 13 - Flat cable of the housing front panel |
| 9 - RAN expander (70 0908 10)                    | 14 - Catch                                 |
|  | 15 - Securing screw                        |

\* Not included in the scope of delivery of the RAN multiplexer Flamengo.

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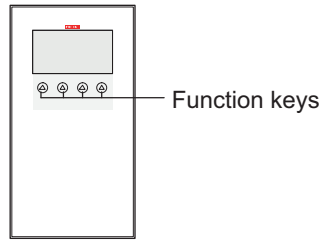
## 8. Configuring the RAN multiplexer Flamengo

### 8.1 Start configuration menu

- Press the left and right function keys simultaneously (3 sec.) until the configuration menu is shown in the display.

#### Symbols in the configuration menu

✓	Checkmark	Set the marked menu item.
▲	Upwards arrow	Scroll upwards through the list.
▼	Downwards arrow	Scroll downwards through the list.
🏠	Home	Cancel, without storing.

### 8.2 Make settings

#### 8.2.1 Setting of configuration menu language

1. Mark „Language“ using the arrow keys. Then press the check-mark.
2. Mark the desired language using the arrow keys: D = German; GB = English etc.
3. Press the check-mark to set the marked language.

#### 8.2.2 Setting of room address

Here you set the same address that was set at old RAN multiplexer (70...) at this location, see chapter 3.2. For general explanation of the room address, see chapter 6.

1. Mark „Address“ using the arrow keys. Then press the check-mark.
2. Mark the desired room address using the arrow keys.
3. Press the check-mark to set the marked address.

#### 8.2.3 Setting of RAN numbers (Room Area Network)

Here you set number of room devices for each connected room. Room devices = Switches, connection sockets, room lamps etc. **No** pear push switch!

1. Mark „RAN“ using the arrow keys. Then press the check-mark.
2. Mark „RAN number“ using the arrow keys. Then press the check-mark.
3. Mark „Room 1“ using the arrow keys. Then press the check-mark.
4. Mark the RAN number for room 1 using the arrow keys.
5. Press the check-mark to set the marked RAN number for room 1.
6. Carry out step 3. to 5. for all connected rooms.

NOTE! Usually, one room lamp corresponds to one RAN user. In the former Tunstall product range, there were room lamps that unusually corresponded to 2 RAN users. This is noted in the documentation for the respective room lamps.

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### 8.2.4 Setting RAN number for collective display

If a room lamp is set to channel number 0, i.e. it is used as a collective display, you must set the RAN number for the collective display:

1. Mark „RAN“ using the arrow keys. Then press the check-mark.
2. Mark „RAN number“ using the arrow keys. Then press the check-mark.
3. Mark „Collective display“ using the arrow keys. Then press the check-mark.
4. Mark the correct value using the arrow keys:  
0 = No collective display installed.  
1 = Collective display installed. The room lamp corresponds to one RAN user.  
2 = Collective display installed. The room lamp corresponds to two RAN users.

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5. Press the check-mark to set the marked value.

### 8.3 Displaying the software revision

If you want to know the RAN multiplexer's software revision:

1. Mark „Info“ using the arrow keys. Then press the check-mark.

The RAN multiplexer's software revision is displayed.

2. Press the home key to return to the menu.

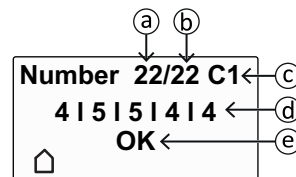
### 8.4 Checking the room bus RAN

#### 8.4.1 Status

Check whether the set RAN numbers for room devices and collective display equal to the numbers of operational devices, see “8.2.3 Setting of RAN numbers” and “8.2.4 Setting RAN number for collective display”.

1. Mark „RAN“ using the arrow keys. Then press the check-mark.
2. Mark „Status“ using the arrow keys. Then press the check-mark.

NOTE! The Test can take some time.

- (a) Sum of all operational room devices
- (b) Sum of all set room devices
- (c) C1: Collective display (1 RAN user) operational  
C2: Collective display (2 RAN users) operational
- (d) Operational room devices per room  
Room 1 | Room 2 | Room 3 | Room 4
- (e) „OK“ : All set RAN numbers (room devices, collective display) match to the numbers of operational devices.  
„Error“: **Not** all set RAN numbers (room devices, collective display) match to the numbers of operational devices! Fix error.

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© Tunstall GmbH, Orkotten 66, 48291 Teigte, Germany, www.tunstall.de