Flurdisplay^{IP} Alpha 16, doppelseitig, Best.-Nr. 76 0160 00

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Corridor display^{IP} Alpha 16, double-sided, order no. 76 0160 00

1. Intended use

The corridor display^{IP} is designed for connection to the IP network of a Flamenco^{IP+} nurse call system. Power is supplied either via PoE+ or via the 24 V DC power supply of the nurse call system. The corridor display^{IP} is used for displaying calls and for the transmission of announcements.

NOTE! The complete installation of the system is described in the Technical Manual.

2. Product description (→ fig. C)

The corridor display^{IP} shows the call type and room number of the call location alphanumerically. The display has 16 digits; longer texts are displayed as a ticker.

The calls are also signalled acoustically. The three call categories (calls, emergency calls, alarms) differ in their tone sequence.

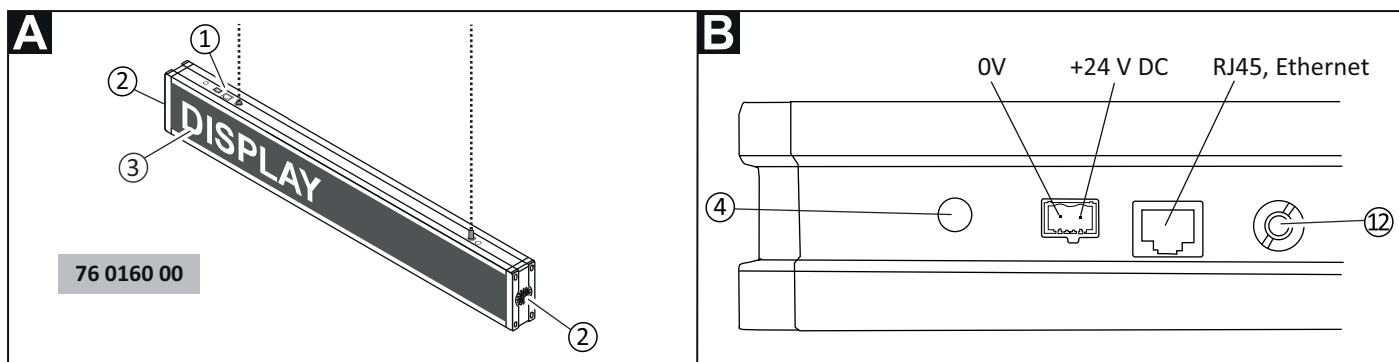
The acoustic call signalling as well as announcements are transmitted via two integrated loudspeakers.

In standby mode, the time is displayed (factory setting).

The corridor display^{IP} is configured with the SystemOrganizer software. Three functions can be disabled on the corridor display^{IP} itself.

3. Mounting location requirements

- Unobstructed view of the display must be ensured at a distance of up to 20 m.
- A distance of at least 50 cm to the wall must be maintained on the left and right of the corridor display^{IP}, because loudspeakers [2] are built in both side housing covers.
- The readability of the display must not be impaired by external light sources.
- Note the position of the ambient brightness sensor [3]! The sensor is used to automatically adjust the brightness of the display to the ambient brightness. A Sensor [3] is only present on the side of the corridor display^{IP} shown in fig. A.
- Mounting height above floor level = 150 – 250 cm.

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- [1] Connection sockets
[2] Two loudspeakers
[3] Ambient brightness sensor
[4] (2 mm under protective cap)
[12] Wire rope holder

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4. Ceiling suspension (→ fig. C – F)

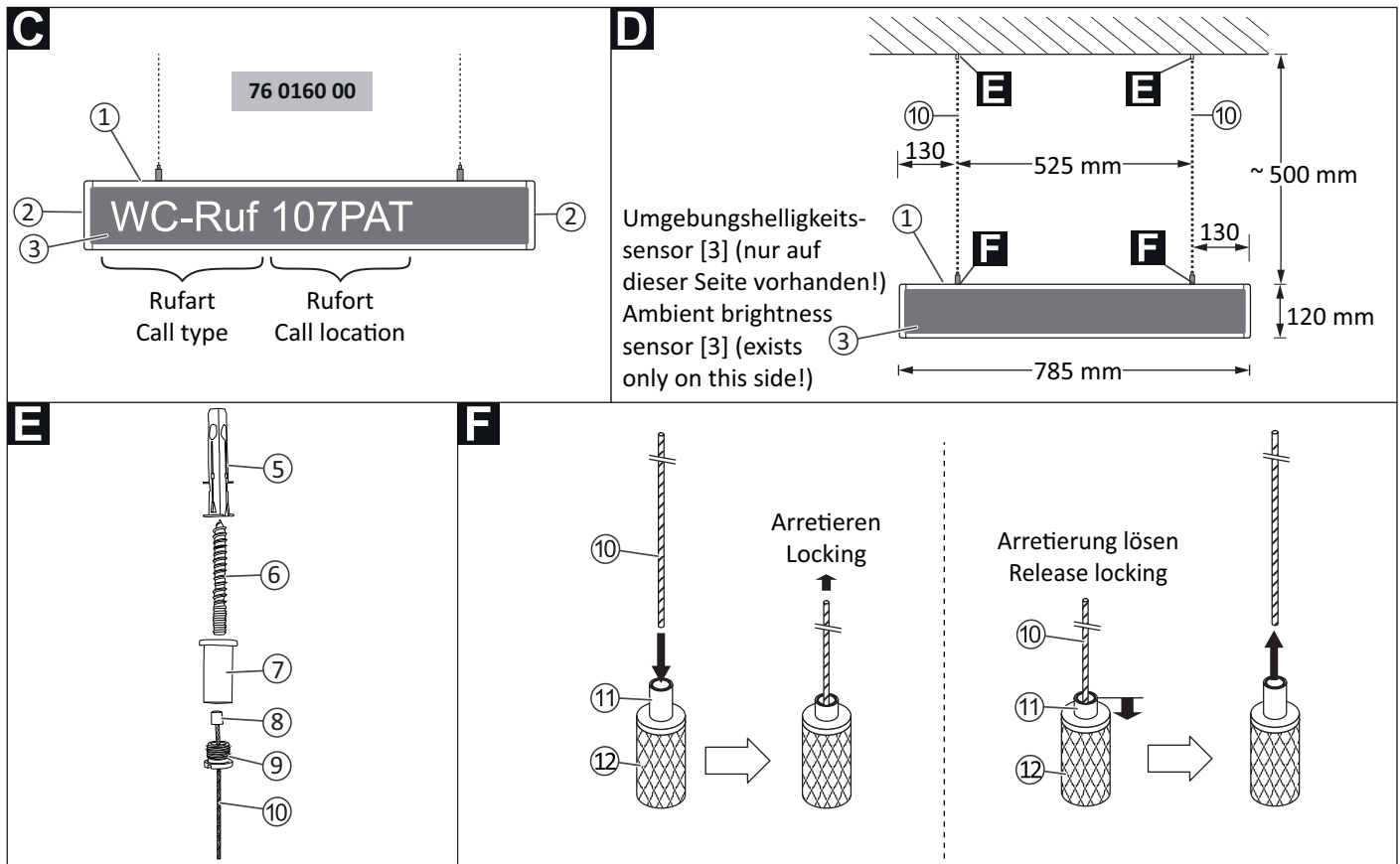
1. Drill two holes in the ceiling (distance: 525 mm). Insert the dowels [5].
2. Each wire rope: Bolt together the wire rope [10] with fitting [8], ceiling fastener [7], screw cap [9] and hanger bolt [6] according to fig. E.
3. Each wire rope: Slide the wire rope [10] into the wire rope holder [12] up to the desired position. To lock the wire rope [10], pull it. A gripping mechanism is activated. Fig. F.

NOTE! To release an inadvertently activated locking, press the head element [11] of the wire rope holder [12]. The gripping mechanism will be released.

4. Screw the two hanger bolts [6] of the pre-mounted corridor display^{IP} into the dowels [5] in the ceiling.

Optional accessories

Extension set ceiling suspension, order no. 19 0780 05, to extend the ceiling suspension by 50 cm.

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- | | |
|-------------------------------|---|
| [1] Connection sockets | [8] Fitting |
| [2] Two loudspeakers | [9] Screw cap |
| [3] Ambient brightness sensor | [10] Wire rope (approx. 500 mm) |
| [5] Dowel (Ø 8 mm) | [11] Head element of the wire rope holder |
| [6] Hanger bolt | [12] Wire rope holder |
| [7] Ceiling fastener | |

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6. Connecting (→ fig. G)

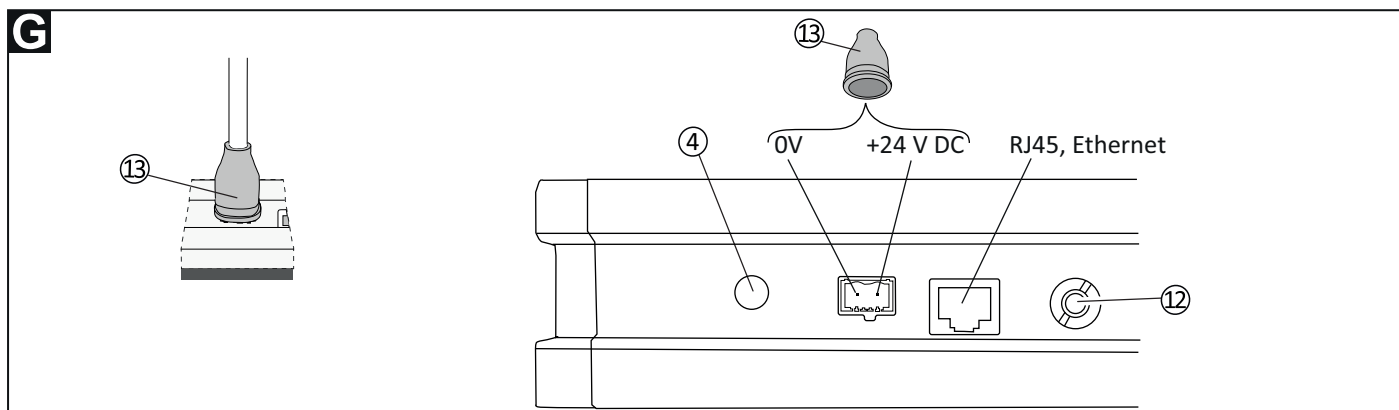
IP network for the nurse call system

- Plug the RJ45 plug of the patch cable into the RJ45 socket of the corridor display^{IP}.

24 V DC power supply

If power is not supplied by PoE+: Connect the corridor display^{IP} to the 24 V DC power supply ring of the ward or directly to a 24 V DC power supply unit via the 2-pole connector:

1. Pull the ESD grommet [13] over the connection cable.
2. Connect the wires to the connector according to fig. G.
3. Plug the connector into the socket on the corridor display^{IP}.
4. Push the ESD grommet [13] over the connector.

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7. Start-up of the unit

When the corridor display^{IP} has been connected to the power supply, it starts up. Sequence of indications in the display: Hardware revision > Firmware revision > All LEDs light up. > Setting* of the function “call tone” (Call vol.) > Setting* of the function “announcements” (Call vol.) > Setting* of the function “time display” (Time) > MAC address.

*: “auto” = Enabled. “off” = Disabled.

At the end of the start-up, the display shows one of the following indications.

- **Time.** Meaning: Corridor display^{IP} is in operation.
- **LED at the bottom left of the display is flashing.** Meaning: Corridor display^{IP} is in operation. “Time display” is disabled.

MAC address of the unit. ERROR! Corridor display^{IP} is not operational. Corridor display^{IP} is not connected to the IP-SystemManager control unit. Correct the error!

8. Functions that can be disabled (option)

The following main functions are active in the factory settings of the corridor display^{IP}, but can be disabled. Disabling the functions is done in the system menu, see chapter 9.

Call tone (Call vol.)

The “call tone” function, i.e. the acoustic signalling of calls shown in the display may only be disabled, if it is ensured that staff on duty are made aware of calls by other signalling.

Announcements (Ann. vol.)

The “announcements” function, i.e. the transmission of announcements may only be disabled, if it is ensured that the reception of the announcements is not required in the vicinity of the corridor display^{IP}.

Time display (Time)

If the display of the time is disabled, an LED flashes at the bottom left of the display in standby mode to indicate that the corridor display^{IP} is in operation.

9. Using the system menu (option)

The system menu serves to:

- display the revisions and IP address of the unit
- test the loudspeakers and the LEDs in the display.
- disable the functions that can be disabled

NOTE! Before using the system menu, be sure to read chapter “8. Functions that can be disabled”.

This is how you use the System menu:

1. Remove the protective cap from the programming button [4], see fig. D.
2. Press and hold the programming button for 10 seconds, e.g. with a small screwdriver or pen.
 - ✓ The display shows: “System-Menu”.
3. Press the programming button briefly.
 - ✓ Display shows hardware revision, e.g. “Rev. A2”.
4. Press the programming button briefly.
 - ✓ Display shows firmware revision, e.g. “FW R1.02”.

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EN - Installation Instructions

5. Press the programming button briefly.
 - ✓ The display shows the own IP address, e.g. “IP: 172.40.2.5”.
 6. Press the programming button briefly.
 - ✓ Loudspeakers are tested by successively emitting a soft, a medium and a loud tone.
 7. Press the programming button briefly.
 - ✓ All LEDs are tested by lighting up.
 8. Press the programming button briefly.
 - ✓ Display shows the setting of the “call tone” (Call vol.) function: “auto” = Enabled. “off” = Disabled.
 9. Set the desired option: Press the programming button briefly to switch between “auto” and “off”.
 10. Press and hold the programming button for approx. 1.5 seconds.
 - ✓ Display shows the setting of the “announcements” (Ann. vol.) function: “auto” = Enabled. “off” = Disabled.
 11. Set the desired option: Press the programming button briefly to switch between “auto” and “off”.
 12. Press and hold the programming button approx. 1.5 seconds.
 - ✓ Display shows the setting of the “Time display” (Time) function: “auto” = Enabled. “off” = Disabled.
 13. Set the desired option: Press the programming button briefly to switch between “auto” and “off”.
 14. To close the system menu and save the settings, press and hold the programming button for 3 seconds.
 - ✓ The system menu is closed. The settings are saved.
 15. Replace the protective cap of the programming button [4].
- NOTE!** You can close the system menu at any menu item and save the settings by pressing and holding the programming button for 3 seconds.

Perform function tests

If you have used the System menu, you must check that all three functions that can be disabled are set as desired: Call tone, Transmission of announcements, time display.

10. Technical data

Standby current consumption	140 mA at 24 V DC
Current consumption on call	450 mA at 24 V DC
Standby power consumption	4.3 W at PoE+
Power consumption on call	13.0 W at PoE+
Dimensions (HxWxD)	125 x 785 x 55 mm
Character height	50 mm
Weight	approx. 2500 g
Housing material	Aluminium, lacquered, with end caps made of ABS, lacquered
Front windows material	Acrylic glass
Degree of protection	IP20
Ambient temperature	+5°C – +40°C
Relative humidity	0 % – 85 % (non-condensing)

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- | | |
|-------------------------------|---|
| [1] Connection sockets | [8] Fitting |
| [2] Two loudspeakers | [9] Screw cap |
| [3] Ambient brightness sensor | [10] Wire rope (approx. 500 mm) |
| [5] Dowel (∅ 8 mm) | [11] Head element of the wire rope holder |
| [6] Hanger bolt | [12] Wire rope holder |
| [7] Ceiling fastener | |

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5. Dismantling (→ fig. F)

1. Disconnect the connection plugs.
2. A second person must hold the corridor display^{IP}.
3. Release the lockings of the two ropes one after the other by pressing down the respective head element [11].

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Plug the RJ45 plug of the patch cable into the RJ45 socket of the corridor display^{IP}.

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7. Start-up of the unit

When the corridor display^{IP} has been connected to the power supply, it starts up. Sequence of indications in the display: Hardware revision > Firmware revision > All LEDs light up. > Setting* of the function "call tone" (Call vol.) > Setting* of the function "announcements" (Call vol.) > Setting* of the function "time display" (Time) > MAC address.

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- **Time.** Meaning: Corridor display^{IP} is in operation.
- **LED at the bottom left of the display is flashing.** Meaning: Corridor display^{IP} is in operation. "Time display" is disabled.
- **MAC address of the unit.** ERROR! Corridor display^{IP} is not operational. Corridor display^{IP} is not connected to the IP-SystemManager control unit. Correct the error!

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Call tone (Call vol.)

The "call tone" function, i.e. the acoustic signalling of calls shown in the display may only be disabled, if it is ensured that staff on duty are made aware of calls by other signalling.

Announcements (Ann. vol.)

The "announcements" function, i.e. the transmission of announcements may only be disabled, if it is ensured that the reception of the announcements is not required in the vicinity of the corridor display^{IP}.

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NOTE! Before using the system menu, be sure to read chapter "8. Functions that can be disabled".

This is how you use the System menu:

1. Remove the protective cap from the programming button [4], see fig. D.
2. Press and hold the programming button for 10 seconds, e.g. with a small screwdriver or pen.
 - ✓ The display shows: "System-Menu".
3. Press the programming button briefly.
 - ✓ Display shows hardware revision, e.g. "Rev. A2".
4. Press the programming button briefly.
 - ✓ Display shows firmware revision, e.g. "FW R1.02".

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EN - Installation Instructions

5. Press the programming button briefly.
 - ✓ The display shows the own IP address, e.g. "IP: 172.40.2.5".
 6. Press the programming button briefly.
 - ✓ Loudspeakers are tested by successively emitting a soft, a medium and a loud tone.
 7. Press the programming button briefly.
 - ✓ All LEDs are tested by lighting up.
 8. Press the programming button briefly.
 - ✓ Display shows the setting of the "call tone" (Call vol.) function: "auto" = Enabled. "off" = Disabled.
 9. Set the desired option: Press the programming button briefly to switch between "auto" and "off".
 10. Press and hold the programming button for approx. 1.5 seconds.
 - ✓ Display shows the setting of the "announcements" (Ann. vol.) function: "auto" = Enabled. "off" = Disabled.
 11. Set the desired option: Press the programming button briefly to switch between "auto" and "off".
 12. Press and hold the programming button approx. 1.5 seconds.
 - ✓ Display shows the setting of the "Time display" (Time) function: "auto" = Enabled. "off" = Disabled.
 13. Set the desired option: Press the programming button briefly to switch between "auto" and "off".
 14. To close the system menu and save the settings, press and hold the programming button for 3 seconds.
 - ✓ The system menu is closed. The settings are saved.
 15. Replace the protective cap of the programming button [4].
- NOTE!** You can close the system menu at any menu item and save the settings by pressing and holding the programming button for 3 seconds.

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