Management Interface

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Management Interface

Interface for connection to the group bus of a CONCENTO^{CARE} nurse call system offers interfaces for:

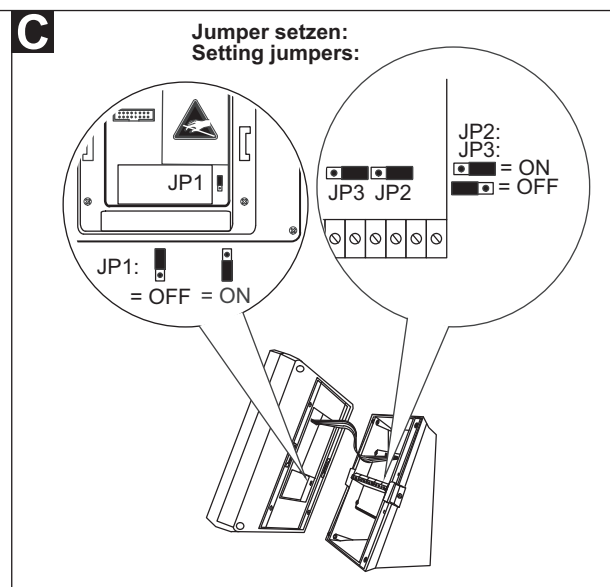
- PC with management software via LAN for configuration and logging.
- DECT system or radio paging system (ESPA 4.4.4) for the transmission of call and system messages.
- Analogue telephone network (a/b interface) for handling of calls per telephone via speech messages. In speech communication systems also for speech communication.
- Fault message facilities (fault message relay output).

NOTE! Only 1 management interface or system interface LAN can be connected to the group bus.

NOTE! The complete installation of the system is described in the Technical Manual.

A Product description

- [1] Control panel
[2] Flat ribbon cable
[3] Base

- [4] Retaining clip with integrated rotary lock
[5] connection field in the base

B Dismantling and disconnecting

At the point of delivery the management interface is assembled and must first be dismantled:

1. Pull off the plugs that are connected to the female connectors on the underside of the management interface.
2. Turn the rotary locks [4] at the right and left on the base [3] to the "open" position with a screwdriver.
3. Hold the control panel [1] with both hands and simultaneously press in both retaining clips [4].
4. **CAUTION!** The control panel [1] and the base [3] are joined by means of a flat ribbon cable [2]. Carefully remove the control panel [1] from the base [3].
5. Pull the flat ribbon cable [2] from the base [3].
6. Disconnect the wires in the connection field [5] of the base [3] (screw-type terminals).

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B Mounting

Wall mounting on a one-gang back box or on cable duct:


1. Set the jumpers (see section “Setting jumpers”).
2. Prepare the wires for connection to the connection field [5] in the base [3] according to section “Connection”.
3. Mount the base [3] to the wall on the four mounting holes with screws and dowels.
4. Feed the connection lines through the break-out in the base [3].
5. Connect the wires to the connection terminals in the base [3] according to section “Connection”.
6. Connect the flat ribbon cable [2].
7. Press the control panel [1] onto the base [3] until it latches in audibly.
8. Turn both rotary locks [4] to the “locked” position with a screwdriver.
9. Connect the connection cables with plug to the female connectors on the underside of the management interface according to section “Connection”.

Before the management interface is ready for operation it must be configured, see the following page.

C Setting jumpers

- Set the jumpers according to the following overview:

Nr.	Position	Setting
JP1	Rear side of the control panel	During operation JP1 must always be set on ON. This activates a lithium battery. It is used to supply the internal clock in the event of a failure of the 24V DC power supply. Factory setting: OFF.
JP2	Connection field	Do not change factory setting ON.
JP3	Connection field	Bus termination: If the management interface is the first or last subscriber on the group bus, the bus must be terminated. For this, set JP3 on ON. No bus termination: Setting OFF. Factory setting: ON.

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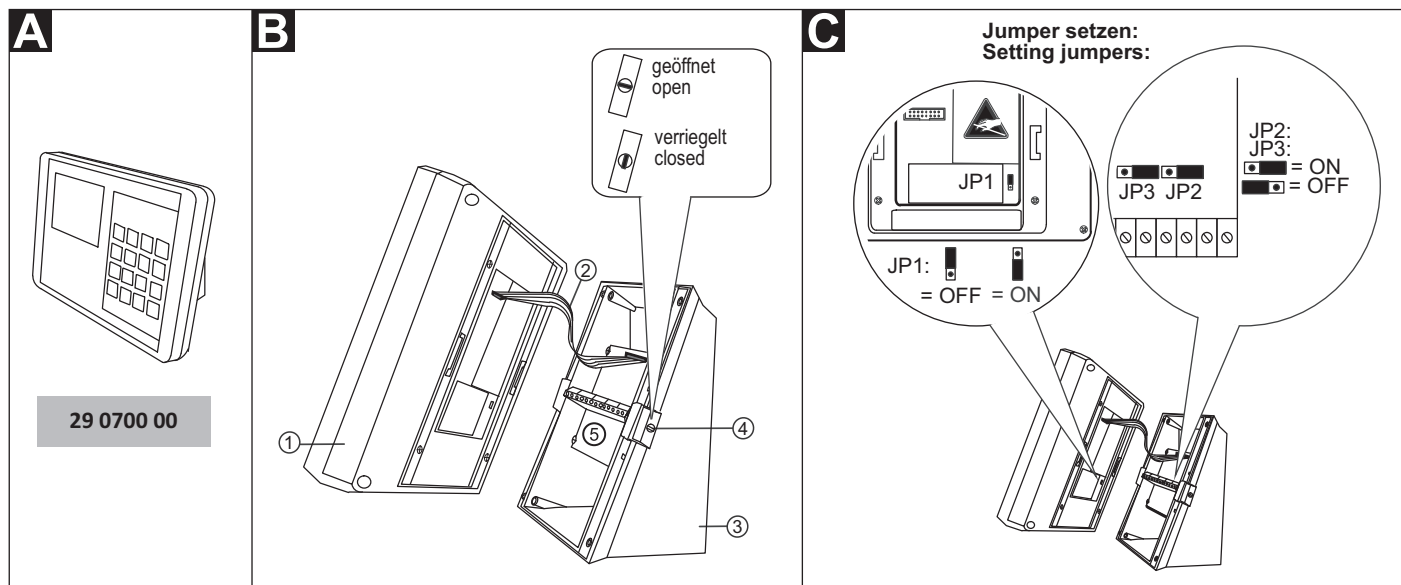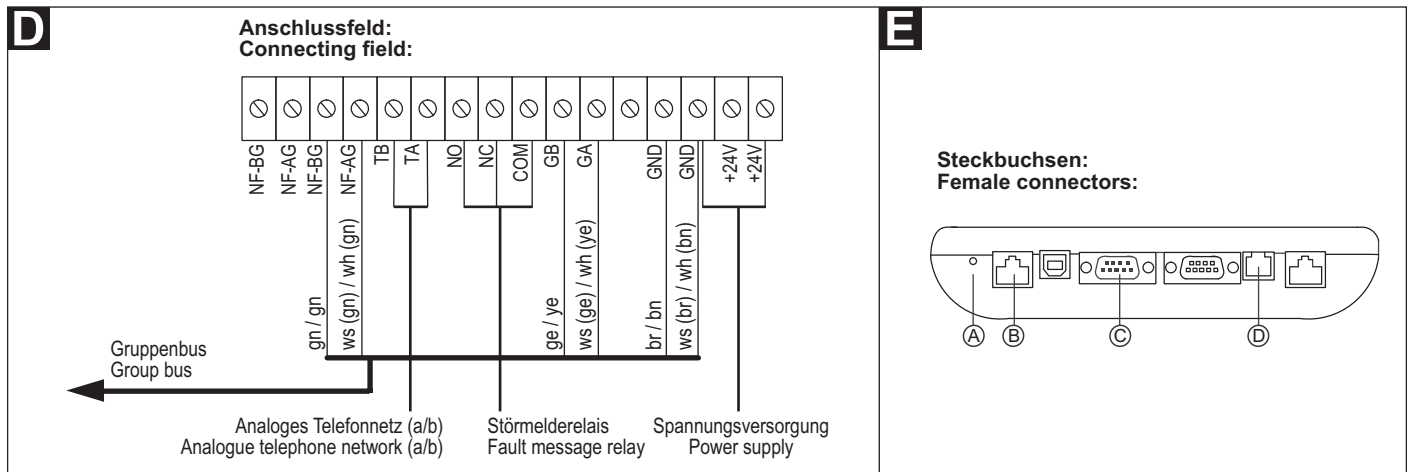
- [1] Control panel
- [2] Flat ribbon cable
- [3] Base

- [4] Retaining clip with integrated rotary lock
- [5] connection field in the base

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D connection field (in the base):		
Group bus	NF-AG and NF-BG are not used in systems without speech communication.	
Power Supply	Connection to the power supply of a physical group. Take current consumption into consideration!	
Analogue telephone network (a/b)	Connection to an RJ11 socket [D] is also possible. However, only one option for connection is permitted to be used!	
Fault message relay	Potential-free. Switching capacity: 2 A. Switches for 2 seconds when: Fault on the group bus, fault on individual modules of the management interface.	
E Female connectors (underside of device):		
A	Reset button	-
B	Management software	RJ45 socket for connection to the LAN with the management software.
C	Radio paging system or DECT (ESPA 4.4.4).	Sub-D plug, 9-pole (RS232). Maximum cable length: 10 m.
D	Analogue telephone network (a/b)	RJ11 socket.

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D Connection of lines to the connection field in the base

1. Strip the connection cables to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
2. Connect the wires to the connection terminals (screw-type terminals) according to the connection diagram.

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E Connection of lines to the female connectors at the underside of the device

1. After the wires have been connected to the connection field in the base of the device and the housing has been locked, insert the plugs at the underside of the device according to the connection diagram.

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Management Interface - Configuration

The following settings must be made on the management interface:

- Setting the IP address,
- Configuring interfaces.

These settings are described in the following.

All other settings are carried out with the management software per remote configuration via the bus system and are not part of this document.

i NOTE! As long as the remote configuration has not been carried out, the display shows at the end of line “ID”: “-> ???”. The management interface is not yet ready for operation.

Displays

Time / Date	Takeover from management software.
Time zone	Active time zone of nurse call system.
Time zone mode	Automatic or manual time zone setting. Configuration in management software.
Active messages	Number of calls, presences and faults in the nurse call system.
ID	Device ID of the management interface. Factory setting.
->???	Management interface has not been configured with the management software!
Add	Address of management interface (setting in management software)
LGR	Logic group of management interface (setting in management software)
PGR	Physical group of management interface (setting in management software)
IP/Subnet/ Gateway	IP configuration of management interface in LAN with management software. Setting on management interface.

Icon displays

Icons display the status of the modules of the management interface.

- Bus = Connection to the group bus
- Server = Connection to management software
- Tel. = Connection to the analogue telephone network
- PSA = Connection to the paging system or DECT system

	Green	OK! Module in management interface is switched on and the connection is ready for operation.
	Red	Fault! Module in management interface is switched on, but the connection is interrupted or faulty.
	Blue	Module in the management interface is switched off.

Use of the service menu

The configurations are carried out in the service menu.

Calling up the service menu:

1. Press **Enter** in the operating display.
 - ✓ “Password” is displayed.
2. Enter password. (Factory setting, Password = 0000).
3. Press Enter.
 - ✓ The main menu appears.

The Management Interface buttons have in the service menu the functions shown in the following figure.

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Buttons on the Management Interface

	Scroll up the list.
	Scroll down the list.
	Select the marked menu item.
	Return to the parent menu. When pressing the main menu simultaneous request to save all changes made.

Saving changes and quitting the service menu

The settings described in the following are saved only after quitting the service menu.

1. Press **Clear** in the main menu.
- ✓ The following message will be displayed: “Save changes? Press 'Enter' to save. Press any key to cancel”.
2. To save the changes, press **Enter**.

Note! If no button is pressed within a period of 120 seconds, the service menu is terminated automatically and the changes are not saved.

Changing the language of the service menu

1. Select “Device settings” in the main menu.
- ✓ Menu “Device settings” is displayed.
2. Select “Language”.
- ✓ The sub-menu “Language” is displayed.
3. Select the desired language.

Setting the IP configuration

The management interface is connected with the management software via LAN. For this, the IP configuration must be set.

1. Select “Device settings” in the main menu.
- ✓ Menu “Device settings” is displayed.
2. Select “IP configuration”.
- ✓ The sub-menu “IP configuration” is displayed.
3. Select “IP address” with **Enter**.
4. Enter the IP address in three-digit format.
Factory setting: 192.168.178.041
Complete the entry with **Enter**.
5. Move to “Netmask” and select with **Enter**.
6. Enter the subnet mask in three-digit format.
Factory setting: 255.255.255.000
Complete the entry with **Enter**.
7. Move to “Gateway” and select with **Enter**.
8. Enter the Gateway in three-digit format.
Factory setting: 192.168.178.001
Complete the entry with **Enter**.
9. Move to “Server Port” and select with **Enter**.
10. Enter the Server Port. Factory setting: 55005
Complete the entry with **Enter**.
11. Press **Clear** as often as required until the following message is displayed: “Save changes? Press 'Enter' to save. Press any key to cancel”.
12. Press **Enter** to save the set IP configuration.
- ✓ The IP configuration will be saved and the service menu terminated.
13. If a new IP address has been set, a reset of the management interface must be carried out. Press the reset button [A] on the left next to the female connectors on the underside of the management interface, see the fig. E.
- ✓ The management interface is restarted and can then be reached from the management software via LAN.

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Setting the time and date

1. Select “Time / Date” in the main menu.
- ✓ The “Time / Date” menu is displayed.
2. Select “Set Time”.
- ✓ The digit to be entered is flashing.
3. Enter the current time.
4. Select “Set Date”.
- ✓ The digit to be entered is flashing.
5. Enter the current date using the format yyyy.mm.dd.
6. To quit the “Time / Date” menu, press **Clear**.

NOTE! The time set here will be overwritten by the management software.

NOTE! The “Auto. daylight saving conversion” feature is currently not used. However, the daylight saving conversion is automatically performed by the management software.

Testing the fault message relay

If a device has been connected to the fault message relay, test the function as follows:

1. Select “System test” in the main menu.
- ✓ The “System test” menu is displayed.
2. Select “Test Fault Relay”.
- ✓ The fault message relay is switched.
3. To quit the test, press **Enter**.

Configuring the paging module (ESPA 4.4.4)

If a DECT system or a radio paging system has been connected to the management interface, proceed as follows:

1. Select “Paging module” in the main menu.
- ✓ The “Paging module” menu is displayed.
2. Select “Paging module On/Off”.
- ✓ The sub menu “Paging module On/Off” is displayed.
3. Select “On” if a radio paging system or DECT system is connected. (Factory setting: “Off”.)

Sending a test message to a pager or a DECT telephone

1. Select “Test message” in menu “Paging module”.
- ✓ “Pager address:” is displayed.
2. Enter the address of the recipient of the test message.
3. Press **Enter**.
- ✓ The test message “Test message” is sent.
4. To quit the “Paging module” menu, press **Clear**.

Factory setting: 1200 baud, 7 data bits, 2 stop bits, parity even.

Configuring the telephone module

If the management interface is connected to the analogue telephone network, proceed as follows:

1. Select “Telephone module” in the main menu.
- ✓ The “Telephone module” menu is displayed.
2. Select “Telephone module On/Off”.
- ✓ The sub-menu “Telephone module On/Off” is displayed.
3. Select “On”, if the management interface is connected to the analogue telephone network. (Factory setting: “Off”.)

Test call for a telephone

1. Select “Test call” in menu “Telephone module”.
- ✓ “Tele. no. :” is displayed.
2. **Enter** the number of the telephone.
3. Press **Enter**.
- ✓ The telephone is called and a message which is to be carried out is transmitted.
4. The person that has been called must press button “0” before he or she hangs up.
5. To quit the “Telephone module” menu, press **Clear**.

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Configuring the telephone module

If the management interface is connected to the analogue telephone network, proceed as follows:

1. Select “Telephone module” in the main menu.
- ✓ The “Telephone module” menu is displayed.
2. Select “Telephone module On/Off”.
- ✓ The sub-menu “Telephone module On/Off” is displayed.
3. Select “On”, if the management interface is connected to the analogue telephone network. (Factory setting: “Off”.)

Test call for a telephone

1. Select “Test call” in menu “Telephone module”.
- ✓ “Tele. no. :” is displayed.
2. **Enter** the number of the telephone.
3. Press **Enter**.
- ✓ The telephone is called and a message which is to be carried out is transmitted.
4. The person that has been called must press button “0” before he or she hangs up.
5. To quit the “Telephone module” menu, press **Clear**.

Setting the speech format for speech messages on the telephone

For speech messages on the telephone you can set how a call location is to be specified. Factory setting: “Room 1234” (room one thousand two hundred and thirty four), Options: “Room 1-2-3-4” (room one two three four), “Ward 1 Room 2-3-4”, “Ward 1 Room 234”, “Ward 12 Room 3-4”, “Ward 12 Room 34”.

Proceed as follows:

1. Select “Telephone module” in the main menu.
- ✓ The “Telephone module” menu is displayed.
2. Select “Speech format”.
- ✓ Sub-menu “Speech format” is displayed.
3. Select the desired option.

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Battery

There is a lithium button cell battery (3 V, CR2032) in the control panel of the Management Interface. It is used to supply the internal clock in the event of a failure of the 24V DC power supply so that the time and date are retained.

Activate/deactivate battery

Jumper JP1 on the rear side of the control panel is used to activate (ON) and deactivate (OFF) the battery, see fig. C on page 1.

When the Management Interface is taken out of operation, e.g. for storing the unit, the battery should be deactivated, i.e. set jumper JP1 in position OFF.

Replace battery

The battery should be replaced every 2 years by a specialist for nurse call systems. Replacement battery: High-quality Li battery, 3 V, CR2032.

CAUTION! While the Management Interface is disconnected from the power supply, the nurse call system continues to function. However, the connection to the analogue telephone network, ESPA 4.4.4 (radio paging system, DECT) and PC are interrupted. Refer to fig. B on page 1 and proceed as follows:

1. Turn the rotary locks [4] at the right and left on the base [3] to the “open” position with a screwdriver.
 2. Hold the control panel [1] with both hands and simultaneously press in both retaining clips [4].
 3. **CAUTION!** The control panel [1] and the base [3] are joined by means of a flat ribbon cable [2]. Carefully remove the control panel [1] from the base [3].
 4. Pull the flat ribbon cable [2] from the base [3].
- ✓ The control panel [1] is now voltage-free.
5. Unscrew the four outer housing screws of the control panel [1] and open the housing.

CAUTION! The printed circuit board is electrostatically sensitive. Avoid direct touch.

6. Remove the used battery out of the holder using a non-conductive tool.
 7. Insert the new battery into the holder.
 8. Re-close the housing and screw in the four housing screws.
 9. Reconnect the flat ribbon cable [2] so that the unit starts again.
 10. Press the control panel [1] onto the base [3] until it latches in audibly.
 11. Turn both rotary locks [4] to the “locked” position with a screwdriver.
- ✓ Because the power supply to the nurse call system was temporarily interrupted, an incorrect time is shown on the display units.
12. If the Management Interface is connected to the Management Software or to an NTP server, the time is corrected automatically. If not, you must set the time and date again using the service menu of the Management Interface, see Technical Manual.

Dispose of battery

The used battery must be taken to a collection point for used batteries or returned to Tunstall.