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Tunstall

Intended use

ComStation IP (76060650) is intended for plug-in connection to the IP network of a ward within a nurse call system of the Flamenco system family.

ComStation IP is a call handling console for the ward.

Important documents

Flamenco system family, Technical Manual (00881397)

The complete installation of the system is described in the technical manual. The manual can be downloaded from the website <https://www.tunstall.com/en-de/>.

<https://www.tunstall.com/siteassets/de-technical-documentation/00881397>

App help

The ComStation IP app provides help that describes in detail how to operate the device.

User guide (00881504)

A detailed user guide can be downloaded as a PDF from the website <https://www.tunstall.com/en-de/>:

<https://www.tunstall.com/siteassets/de-technical-documentation/00881504.pdf>

Quick reference guide (00881505)


A quick reference guide is included with the ComStation IP and can be downloaded from www.tunstall.de:

<https://www.tunstall.com/siteassets/de-technical-documentation/00881505.pdf>

GRANDSTREAM documentation

The scope of delivery of the ComStation IP includes documents from the device manufacturer GRANDSTREAM. Please keep these documents, as they may be required by technical personnel if needed. These documents are not intended for nursing staff.

Scope of delivery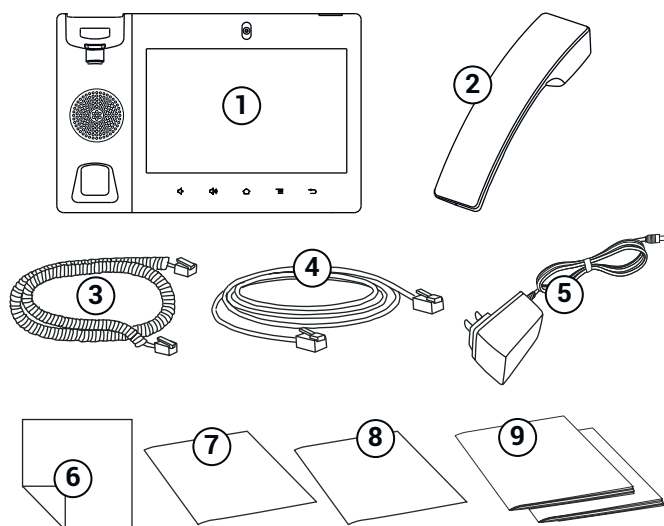

- [1] Main case
- [2] Handset
- [3] Handset cord
- [4] Ethernet cable
- [5] 12 V power adaptor
- [6] Screen cleaning cloth
- [7] Installation Instructions for ComStation IP
- [8] Quick reference guide for ComStation IP
- [9] Documents by the GRANDSTREAM company (for technicians only!)

Misuse

ComStation IP is based on an IP desk phone from the manufacturer GRANDSTREAM with an adapted Android operating system. The call handling functionality is pre-installed as an app. All other features are inactive (kiosk mode).

Please note:

- The IP desk phone may only be used as ComStation IP. No other apps may be installed or used.
- The use of ports on the device for purposes other than the ComStation function is prohibited.
- The IP desk phone must not be connected to the Internet. Only updates from Tunstall may be installed on the IP desk phone.
- The Android operating system must not be reset to factory settings. This would result in the loss of changes to the operating system that can only be made by Tunstall.

Power supply

Power supply options

The device can optionally be supplied with power as follows:

- with the included 12 V power adaptor* (input: 100 – 240 V AC, 50 – 60 Hz; output: 12 V DC, 1,5 A).
- on a LAN with PoE or PoE+ (PoE 802.3af Class 3, PoE+ 802.3at, Class 4).

*) Only use the 12 V DC power adaptor included in the scope of delivery. Using a different power adaptor may damage the device.


Power source for safety purposes

The German standard DIN VDE 0834 stipulates that, in the event of a failure of the general power supply, the nurse call system must be supplied from a power source for safety purposes according to DIN VDE 0100-200 and DIN VDE 0100-560. This means that the 230 AC outlet respectively the PoE/PoE+ network switch or injector must be connected to an UPS/auxiliary power supply.

Location for the device

- 20 cm minimum distance between the device and the user's body.
- The area in front of the microphone (front right) should be kept clear. The optimum distance for hands-free talking is 50 cm.
- Do not place the device in a room with high dust generation, as this can considerably reduce the service life of the device.
- No direct sunlight or other heat radiation, as this can damage the electronic components and the plastic housing.
- Not in environments where steam is generated.


Setting up the device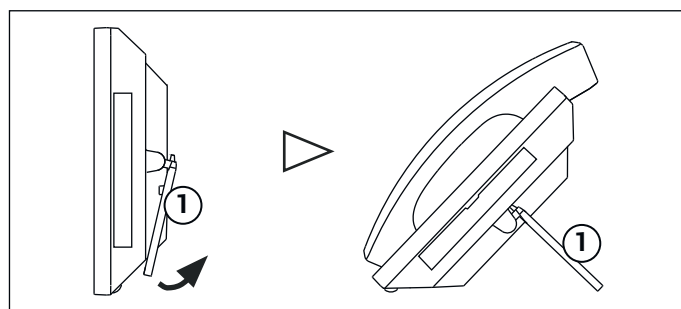

[1] Built-in stand for desktop installation

The device has a built-in stand. To use it, fold it out of the back of the device and adjust it to the desired angle. Ensure that the device is in a secure position.

Optional wall mounting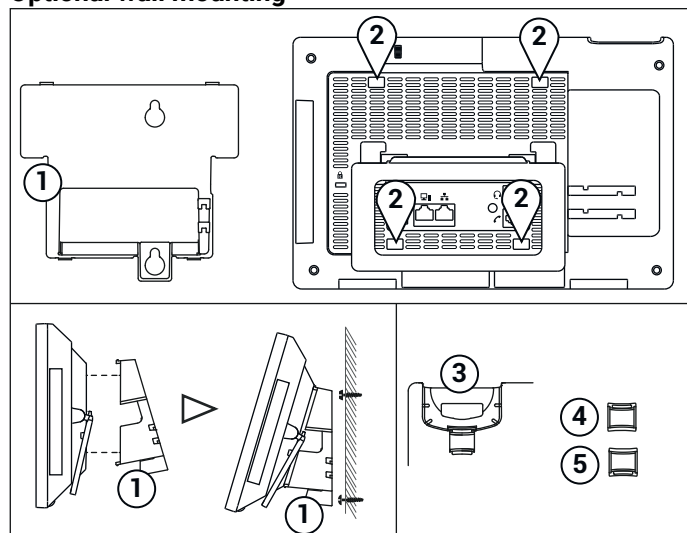


- [1] Wall mount (order separately, article no. 76060660)
- [2] Four mounting slots for attaching the wall mount
- [3] Handset cradle
- [4] Tab in the position for wall mounting (tab with extension up)
- [5] Tab in the position for desktop installation (tab with extension down)

1. Attach the wall mount to the four mounting slots on the back of the device.
2. Attach the device to the wall via the two keyholes in the wall mount.
3. Pull out the tab below the handset cradle. Rotate the tab and plug it back into the slot with the extension up to hold the handset while the device is mounted on the wall.

Connecting the device

Ports

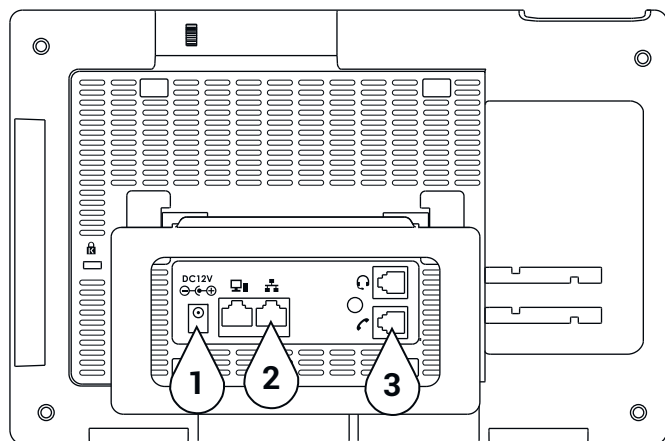- [1] Power jack for 12 V DC power adaptor (only required if the device is not powered via the Ethernet cable (PoE))
- [2] LAN port
- [3] Handset cord connection

Proceed as follows

NOTICE

Prerequisite for the operation of ComStation IP is that the nurse call system has been configured for use with the SystemOrganizer configuration software.

1. Connect the handset cord to the handset and to the port for handset cord.
2. Connect the Ethernet cable to the LAN port and to the RJ45 connection socket in the room. The RJ45 connection socket must be connected to the nurse call system in accordance with the technical manual for the nurse call system.

CAUTION

Make sure that the latch of the RJ plugs engages so that the plugs cannot slip out.

3. If the device is not powered by PoE or PoE+ (PoE network switch), connect the 12 V DC power adaptor to the power jack and to a 230 V socket.

CAUTION

DO NOT switch off the device during system start-up or when updating the firmware. Otherwise, the firmware could be damaged.

The device is starting up. The ComStation IP app login screen appears automatically.

4. Tap **LOG IN**.

You are logged in. The home screen of the ComStation IP App appears. ComStation IP is in operation.

CAUTION

You must secure all plugs (Ethernet cable, 12 V plug power adaptor) against unintentional removal. It is recommended that the RJ45 connection socket and the 230 V socket be marked with a warning: "DANGER! Do not remove the plug! The device is part of the nurse call system".

For information on how to operate the ComStation IP or change default settings, please refer to the App Help. To open the help, tap on the question mark in the home screen.

Functional checks must be carried out as described in the technical manual, before the ComStation IP can be used.

Technical data

Dimensions (HxWxD)	approx. 160 x 260 x 155 mm (incl. handset; set up on stand)
Screen diagonal	8" (1280x800)
Weight	1450 g
Power consumption	<ul style="list-style-type: none"> • via PoE (802.3af/at): up to approx. 12.5 W • via power adaptor (12 V/1.5 A): rated power max. 18 W
Ambient temperature:	
• Transport, storage	• -10 °C to + 60 °C
• Operation	• 0 °C to 40 °C
Relative humidity	10 % to 90 % (non condensing)

Tunstall GmbH, Orkotten 66, 48291 Telgte, Germany
Tel: +49 (0) 2504/701-0, E-Mail: DE.Info@tunstall.com
www.tunstall.de/en-de/

Tunstall