

CONCENTO^{CARE}

User instructions for your nurse call system


To raise a call:

All red buttons or operating elements of your nurse call system are used for raising calls.


If you need help:



- press a red call button [1],

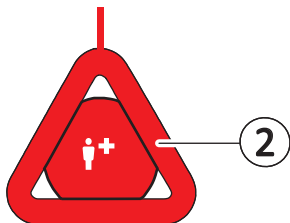


- pull a red call cord [2]

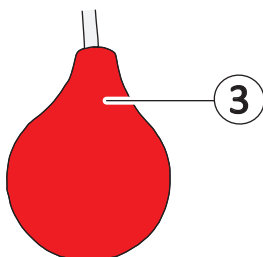


- or squeeze a red call ball [3].

The symbol on the call button [1] lights up brightly to confirm your call.

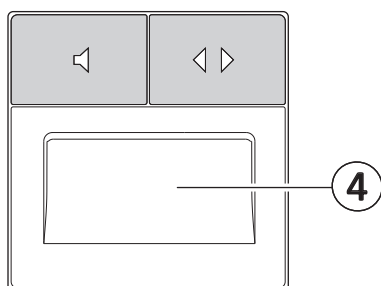

Nursing staff comes to you or speaks to you via the loudspeaker [4].

You can answer directly without pressing another button.

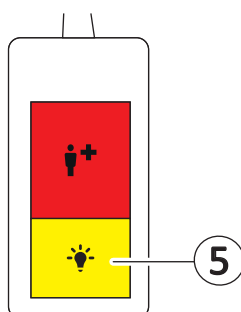

Switching the light on and off:

- Press the yellow light button [5].
In this case, no call is raised!