


Pull cord call switch insert

without frame

29 0707 20F

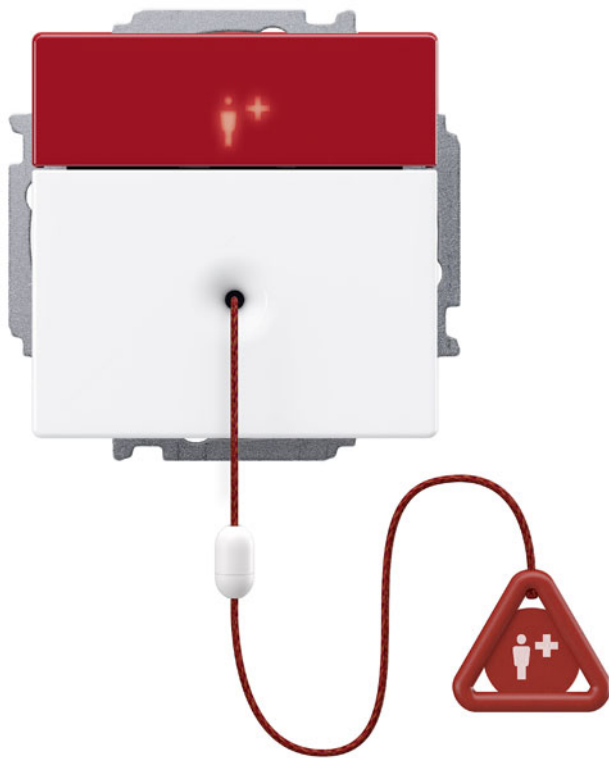

General description

Switch insert with a pull cord at the end of which a call handle is attached, for triggering calls or WC calls by pull actuation. Another call option is a red call button with reassurance light.

Particularly suitable for triggering calls in the shower or next to the WC.

The reassurance light illuminates brightly as soon as a call is raised. If presence in the room is switched on when a call is raised, an emergency call is triggered.

Cancel function for WC calls optionally configurable.

Suitable for wall and ceiling mounting.

Specification


- 2.50 m long pull cord incl. call handle with symbol and safety release
- Safety release in the pull cord opens when the pulling force exceeds a limit value of approx. 25 Nm (strangulation protection). The safety release is closed again by simply plugging it together.
- Red call button with reassurance light and integrated location light
- Acoustic feedback for call activation
- Usable for call (factory setting) or WC call
- The pull cord switch can be configured in such a way that WC calls can not only be raised but also cancelled with activated staff presence using the pull cord (Function available from production week 24/2021). To comply with DIN VDE 0834, an additional cancel switch must be installed.
- A channel number can be set on the pull cord switch to specify the call location in the room in more detail. The channel number can, for example, identify beds or WC cubicles.
- Standard set to channel 0
- Identifiable as part of the nurse call system through design features
- Easy plug-and-play mounting
- Wall mounting or ceiling mounting in 1-gang back box
- Mounting in single frame
- Integrated electronics with 3-pole plug connection for the room bus (RAN)

Pull cord call switch insert

without frame

29 0707 20F

- Page 2 -

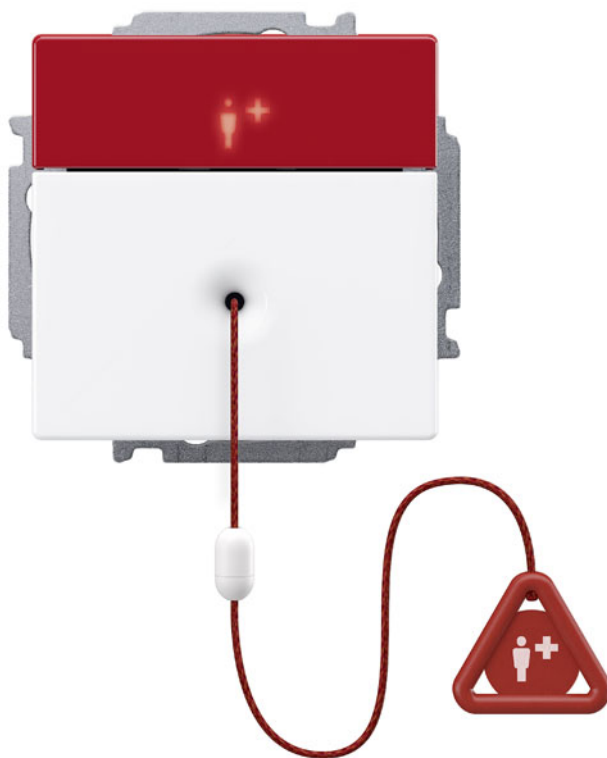

Scope of supply

- Pull cord switch insert
- Mounting plate
- Terminal block, 5-pole, black/yellow/red/light grey/dark grey, plugged in
- Installation instructions

Technical data

Nominal voltage:	24 V DC
Standby current consumption:	9 mA
Degree of protection:	IP22
Dimensions (HxW):	71 x 71 mm
Installation depth:	24 mm
Housing material:	Polycarbonate
Pull cord material:	Polyamide
Call handle material:	ABS
Ambient temperature:	+5 °C – +40 °C
Relative humidity:	0 % – 95 %

Order no.	Product name	Note
29 0707 20F	Pull cord call switch insert	Design future® linear
Frame, order separately:		
77 0210 51	Frame A, 1-gang	
77 0210 52	Frame C, 1-gang	
77 0210 52	Frame F, 1-gang	
Necessary accessories, order separately:		
17 0100 00	Back box solid wall, 1-gang	see appendix "Accessories"
17 5100 00	Back box partition wall, 1-gang	see appendix "Accessories"
Spare parts, reorder if required:		
19 9500 19	Terminal block, 5-pole, bk/ye/rd/gy/gy	see appendix "Accessories"
77 9510 06	Pull cord incl. call handle, safety release	see appendix "Accessories"